

Prior to use, carefully read  
the instructions for use.

GB

# GRADIA DIRECT

## SHADE GUIDE

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

Used for shade selection.

### HOW TO USE

1. Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Keep the teeth hydrated. Select the appropriate GRADIA DIRECT shades by referring to the GRADIA DIRECT shade guide.
2. Each shade tab consists of four levels with 1.0, 1.5, 2.0 and 3.0 mm in thickness respectively. When layered restorations are desired, two or more shade tabs can be placed on top of each other to estimate the resulting layered shades.

### PACKAGE

1 piece

### NOTE

The shade guide can be cleaned with water and mild soap. It should not be sterilized. The use of various chemical disinfectants / sterilants may cause damage to the shade guide.