



Patterson® Etch Gel

Gel de mordançage

38% Phosphoric Acid

Acide phosphorique à 38 %

Patterson® Etch Gel is a thixotropic phosphoric acid etch gel that stays in place and will not run onto exposed tissue. Syringe dispensing through pre-bent small gauge applicator tips offers precise placement of gel even in relatively inaccessible areas. Etch washes off easily with water.

PRODUCT DESCRIPTION

38% phosphoric acid dental etching gel

INDICATIONS

For acid etching dentin, enamel and glass ionomer cement

CONTRAINDICATIONS

Do not place on soft tissues

DIRECTIONS FOR USE

1. Prepare tooth and cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophyl paste. Rinse and dry thoroughly.
2. Securely attach 25-gauge luer lock applicator tip to syringe. Start flow on pad; not in patient's mouth. Do not push on plunger with thumb. For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel.
3. Recommended etching time for enamel is 15-20 seconds and for dentin is 15 seconds. To the extent possible, apply Etch first to enamel and then to dentin. Leave on dentin for 15 seconds. Cap syringe immediately after use.
4. Rinse off Etch Gel with copious amounts of water. Do not disturb this surface.
5. Leave surface wet, or dry surface with clean, uncontaminated air, depending on the instructions for use for the bonding system you are using.

Note: Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds and rinse with copious amounts of water.

WARNING:

If Etch Gel contacts skin or soft tissue, rinse off immediately with copious amounts of water.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature.
- Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient.

For additional information, refer to:

<https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

PRECAUTIONS

Avoid contact with oral tissue, eyes and skin. If accidental contact occurs wash with copious amounts of water.

In case of eye contact, wash eye for 15 minutes and seek medical attention.

Keep out of reach of children.

Do not take internally.

Use at room temperature.

Do not use after expiry date.

Keep syringe tightly capped when not in use.

Caution: Federal Law restricts this device to sale by or on the order of a dentist.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient.

For additional information, refer to:

<http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

Manufactured for:
Fabrique pour:

Patterson Dental Supply, Inc.
1031 Mendota Heights Road
Saint Paul, MN 55120

