

# Impressions

Now available in




Virtual<sup>®</sup> 380

Impression material

ivoclar  
vivadent<sup>®</sup>

# Impressive Impressions

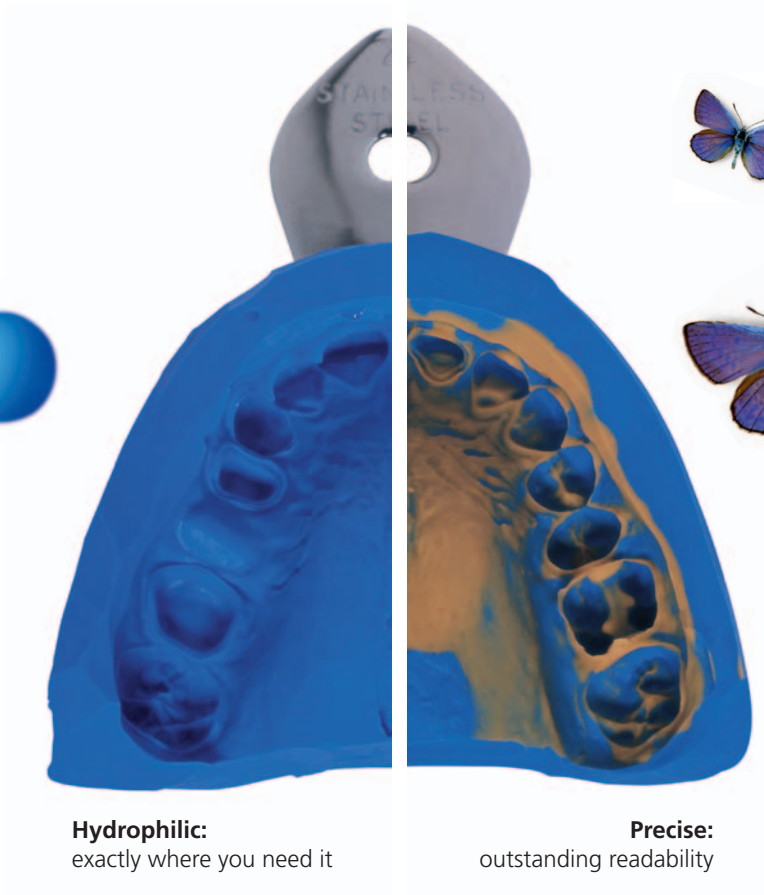

**Hydrophilic:**  
exactly where you need it

**Precise:**  
outstanding readability

## Virtual<sup>®</sup> 380

The hydrophilic precision impression material



# New, XX-tra large delivery form: Virtual® 380 in XXL cartridges

## X-tremely practical:



Insert XXL cartridge into the mixing unit



Dispense material without the mixing tip in place to verify an even flow of both components



Attach the mixing tip



Place the bayonet ring and lock it into position by turning clockwise.

**Virtual is a hydrophilic precision impression material based on vinyl polysiloxanes** (addition-reaction silicones).

Available in both fast and regular setting times, the system features the five viscosities Putty, Heavy Body, Monophase, Light Body and Extra Light Body to suit all popular impression taking techniques.

The Heavy Body and Monophase consistencies continue to be available in the tried-and-tested 50-ml cartridges suitable for use in combination with the hand-held dispenser.

Optionally, the two consistencies are now offered as Virtual 380 in the new, practical XXL cartridges for use in the Pentamix\*, Plug&Press Dispenser\* or MixStar\* mixing units.

\* Not registered trademarks of Ivoclar Vivadent AG

In addition, the Virtual line of products includes the fast-setting Virtual Bite Registration material with a high final hardness. The system is complemented by a full line of accessories, including a tray adhesive.

**Outstanding properties** of the multiple award-winning Virtual impression material:

- hydrophilic exactly where you need it
- very easy to read
- customised to suit the clinical indication
- convenient for both dentist and patient
- balanced in every respect
- dependable in practice tests



The complete Virtual line of products, including the new XXL cartridges

Descriptions and data constitute no warranty of attributes.  
Printed in Liechtenstein  
© Ivoclar Vivadent AG, Schaan/Liechtenstein  
600690/0306/e/BVD

## Delivery forms:

### Virtual 380 Heavy Body Refill Fast Set

- 2 Cartridges base and catalyst, 380 ml each
- 15 Mixing tips
- 2 Bayonet rings

### Refill Regular Set

- 2 Cartridges base and catalyst, 380 ml each
- 15 Mixing tips
- 2 Bayonet rings

### Multi Pack Fast Set

- 6 Cartridges base and catalyst, 380 ml each
- 45 Mixing tips
- 6 Bayonet rings

### Multi Pack Regular Set

- 6 Cartridges base and catalyst, 380 ml each
- 45 Mixing tips
- 6 Bayonet rings

### Virtual 380 Monophase Refill Fast Set

- 2 Cartridges base and catalyst, 380 ml each
- 15 Mixing tips
- 2 Bayonet rings

### Multi Pack Fast Set

- 6 Cartridges base and catalyst, 380 ml ea
- 45 Mixing tips
- 6 Bayonet rings

### Virtual 380 Accessories

- Refill Bayonet Ring
- 2 Bayonet rings

- Refill Mixing Tip
- 50 Mixing tips



Production and Distribution:  
**Ivoclar Vivadent AG Clinical**  
Bendererstrasse 2  
FL-9494 Schaan  
Fürstentum Liechtenstein  
Tel. +423 / 235 35 35  
Fax +423 / 235 33 60  
www.ivoclarvivadent.com

**ivoclar**  
**vivadent®**

# in XXL!

**If you are a user of mixing units, you too can now benefit from the excellent quality of Virtual with the new Virtual 380 XXL cartridge.**

## Virtual 380 Heavy Body



Because of its non-slump consistency and high final hardness, the high-viscosity Virtual 380 Heavy Body material is ideal as a tray material in combination with Virtual Light Body when the putty/wash technique is used. Virtual 380 Heavy Body is offered in two different working and setting times:

	Working time	Time in mouth
Fast Set:	1:15 min.	2:30 min.
Regular Set:	2:05 min.	4:30 min.

## Virtual 380 Monophase



The exceptional elasticity and high final hardness of the medium-viscosity Virtual 380 Monophase makes it perfectly suitable for dental implant impressions, overimpressions and functional impressions. As a result of its outstanding thixotropic behaviour, the material builds up sufficient pressure and at the same time perfectly adapts to primary parts and impression posts.

Virtual 380 Monophase is available in the following working and setting times:

	Working time	Time in mouth
Fast Set:	1:15 min.	3:30 min.

### **X-ceptional benefits**

of the XXL cartridge in conjunction with the mixing unit:

- ready for use in no time
- no special activation required
- more homogeneous mix due to the active mixing process
- large, economical package size
- quick dispensing of ready-mixed material