

Clinpro™ Sealant



So simple, you'll
be tickled pink.



Clinpro™ Sealant

- Convenient direct delivery syringe for easy application
- Color-change technology to visualize placement
- Low viscosity to flow easily into pits and fissures
- Contains and releases fluoride
- Bonds to enamel

An exciting advancement in sealant technology that's hard to overlook.

3M ESPE is committed to providing the most advanced and patient-friendly preventive products to dental professionals and their patients.

Clinpro sealant, along with Clinpro™ Prophy Paste and Clinpro™ Prophy Angles, offer several benefits that make dental work easier for you and your patients.


Clinpro sealant goes on pink for easy-to-use application, and cures to a natural white. Applied with our convenient ultra-fine syringe tip, its low viscosity formula flows easily into pits and fissures.

Dental Professionals prefer it over the other sealants.

Seventy-seven percent of respondents rated the overall performance of Clinpro sealant to be superior to their current sealant.

Performance Rating

Clinpro Sealant vs. Sealant Currently Used

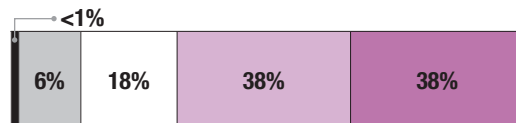

N = 145



Source: 3M ESPE Sealant Evaluation Summary Report, The Market Solutions Group, Inc., February 2001.

Before



After



Photography by Dr. Chris Bryant

Simplified Sealant Technique with Adper™ Prompt™ Self-Etch Adhesive

Adper Prompt self-etch adhesive may be used instead of the acid etch technique when placing Clinpro sealant. Follow the instructions below.

Preparation: Thoroughly clean teeth to remove plaque and debris from enamel surfaces and fissures. Rinse thoroughly with water and dry.

Note: Do not use any cleaning medium that contains oils or fluoride. Prepare adhesive according to dispensing instructions.



1 Brush adhesive onto surface to be sealed; scrub for 15 seconds applying moderate pressure. **Do not shorten this time. Proper bonding will not occur if the solution is simply applied and dispersed.**



2 Use a gentle stream of air to **thoroughly dry** the adhesive to a thin film. A second layer is not required for bonding sealants.



3 Apply a **light-cured** sealant. (Note: Adper Prompt self-etch adhesive is not indicated for self-cure sealants.)



4 Light-cure the sealant and adhesive together with a halogen, LED or plasma light for the amount of time specified to cure the sealant.

Ordering Information



Item Number	Product Information
3M™ ESPE™ Clinpro™ Sealant	
12626	Introductory Kit, Syringes Contains: 2 – 1.2 ml Clinpro Sealant Syringes; 1 – 3 ml 3M™ ESPE™ Scotchbond™ Etchant Syringe; 20 – Sealant Syringe Tips (Black); 25 – Etchant Syringe Tips (Blue)
12627	Refill, Syringe Contains: 1 – 1.2 ml Clinpro Sealant Syringe; 10 – Sealant Syringe Tips (Black)
12621	Introductory Kit, Bottles Contains: 2 – 6 ml Clinpro Sealant Bottles; 1 – 9 ml 3M™ ESPE™ Scotchbond™ Etchant Bottle; 2 – Brush Handles; 60 – Brush Tips; 50 – Fiber Tips; 1 – Mixing Well with Cover
12622	Refill, Bottle Contains: 1 – 6ml Clinpro Sealant Bottle
3M™ ESPE™ Adper™ Prompt™ L-Pop™ Self-Etch Adhesive*	
41923	Mini-Pack Includes 20 L-Pop Applicators with XS Brushes
41925	Refill Pack Includes 40 L-Pop Applicators
41926	Giant Pack Includes 100 L-Pop Applicators
41927	Bulk Pack Includes 200 L-Pop Applicators
41928	Economy Pack Includes 500 L-Pop Applicators

*Also available in vial dispensing

Customer Care Center: 1-800-634-2249 www.3MESPE.com

3M ESPE

Dental Products

3M Center
Building 275-2SE-03
St. Paul, MN 55144-1000
U.S.A.
1-800-634-2249

3M Canada

Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

Please recycle. Printed in U.S.A.
© 3M 2008. All rights reserved.
70-2009-3697-2

3M, ESPE, Adper, Clinpro, L-Pop, Prompt and Scotchbond are trademarks of 3M or 3M ESPE AG. Used under licence in Canada.