# GRADIA® DIRECT

## LIGHT-CURED COMPOSITE RESTORATIVE

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**

**A. GRADIA DIRECT ANTERIOR**

- 1. Direct restorative for Class III, IV, V cavities.
- 2. Direct restorative for wedge-shaped defects and root surface cavities.
- 3. Direct restorative for veneers and diastema closure.

**B. GRADIA DIRECT POSTERIOR (radioopaque)**

- 1. Direct restorative for Class I and II cavities

**CONTRAINDICATIONS**

- 1. Pulp capping.
- 2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**DIRECTIONS FOR USE**

- 1. Shade Selection. Clean tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate GRADIA DIRECT shade by referring to the GRADIA DIRECT shade guide.
- 2. Cavity Preparation. Prepare cavity using standard techniques. Note: For pulp capping, use calcium hydroxide.
- 3. Bonding Treatment. For bonding GRADIA DIRECT to enamel and / or dentin, use a light-cured bonding system such as GC FUJI BOND LC, GC UNIFI® Bond or GC G-BOND™ (Fig. 1). Follow manufacturer's instructions.
- 4. Placement of the GRADIA DIRECT Unit into the Unit Applicator or equivalent. Remove the cap and extrude material directly into the prepared cavity. Use steady pressure (Fig. 2). Maintain pressure on the applicator handle while removing the Unit Applicator and Unit from the mouth.
- 5. Finish the edges the Unit from coming loose from the applicator.
- 6. Dispensing from a syringe. Remove syringe cap and dispense material onto a mixing pad. Place the material into the cavity using a suitable placement instrument. After dispensing, screw syringe plunger anticlockwise by a half to full turn to release residual pressure inside the syringe. Replace cap immediately after use.

1. Basically, material can be applied in a single layer to achieve aesthetic restorations using Standard shades. For details, refer to the Clinical Instructions.

2. Material may be hard to extrude immediately after removing from cold storage. Prior to use, leave to stand for a few minutes at normal room temperature.

3. After dispensing, avoid too long an exposure to ambient light. Ambient light can shorten the manipulation time.

**Clinical Hints**

a. In the case of small cavities Restoring using a one shade layering technique. In most cases the use of one Standard shade alone will be sufficient. In cases where a higher degree of translucency is needed, one of the Outside special shades can be selected. See also Examples of Clinical Applications.

b. In the case of large cavities In most cases a multi shade layering technique will give the best aesthetic results. To block out shine throughs from the oral cavity to mask discoloured dentin, select an appropriate Inside special shade and continue to build up with a Standard shade.

To make a restoration more life like e.g. to copy age related changes in appearance, the final layer should consist of an Outside special shade. See also Examples of Clinical Applications and / or consult the Shade Combination Chart.


**2. Posterior cavities**

a. Clean case of small cavities Restore using a one shade layering technique. In most cases the use of one Standard shade alone will be sufficient. In cases where a higher translucency is needed, one of the Outside special shades can be selected. See also Examples of Clinical Applications.

b. In the case of deep cavities Place a flowable composite such as GRADIA DIRECT Flo or GRADIA DIRECT LoFo™ on the cavity first. Then place a Standard shade. For optimal aesthetics use an Outside special shade as the final composite layer. See also Examples of Clinical Applications.

\*GC FUJI LINING® PASTE PAK, GC FUJI LINING® LC or GC FUJI IX GP can also be used as a liner or base material. Follow the respective manufacturer's instructions for use.

### Examples of Clinical Applications (Clinical Hint No.1, 2)

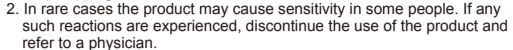### Shade combination chart for multiple layers in large cavities of anterior teeth (1-8)

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For details of shades, refer to the following section of SHADES.

- 5. Contouring before Light Curing. Contour using standard techniques.
- 6. Light Cure GRADIA DIRECT using a light curing unit (Fig. 3). Refer to the following chart for Irradiation Time and Effective Depth of Cure.

### GRADIA DIRECT ANTERIOR: Irradiation Time and Effective Depth of Cure

Irradiation time	Plasma arc (2000mW/cm²)	3 sec.	6 sec.
GC G-Light (1200mW/cm²)	10 sec.	20 sec.	30 sec.
Halogen / LED (700mW/cm²)	20 sec.	40 sec.	40 sec.
CT, NT, WT, GT, CVT			
A1, A2, B1, B2, XBW, BW, DT	2.5 mm	3.0 mm	3.0 mm
A3, B3	2.0 mm	3.0 mm	3.0 mm
A3.5, A4, C3, A02, A03, A04, CV, CVD	1.5 mm	2.5 mm	2.5 mm

### GRADIA DIRECT POSTERIOR : Irradiation Time and Effective Depth of Cure

Irradiation time	Plasma arc (2000mW/cm²)	3 sec.	6 sec.
GC G-Light (1200mW/cm²)	10 sec.	20 sec.	30 sec.
Halogen / LED (700mW/cm²)	20 sec.	40 sec.	40 sec.
P-A1, P-A2, P-WT, P-NT			
P-A1, P-A2, P-WT, P-NT	2.5 mm	3.0 mm	3.0 mm
P-A3, P-A3.5	2.0 mm	3.0 mm	3.0 mm

Note:

- 1. Material should be placed and light cured in layers. For maximum layer thickness, please consult the tables.
- 2. Lower light intensity may cause insufficient curing or discoloration of the material.

7. Finishing and polishing. Finish and polish using diamond burs, polishing points and discs. To obtain a high gloss, polishing pastes can be used.

**Cleaning and Sterilizing of the Unit Applicator**

- 1. Clean applicator with gauze and alcohol.
- 2. The applicator may be steam autoclaved at 121-135°C (250-275°F) for 10 to 30 minutes or disinfected using alcohol.

Note:

- 1. Do not attempt to sterilize in a unit at a temperature that exceeds 135°C (275°F) during the drying step of the autoclave procedure.
- Otherwise, the applicator may be damaged.
- 2) The use of various chemical disinfectants / sterilants may cause damage to the applicator and therefore is not recommended.

**Cleaning of the Shade Guide**

The shade guide can be cleaned with water and mild soap. It should not be sterilized. The use of various chemical disinfectants / sterilants may cause damage to the shade guide.

**SHADES**

1.22 shades for anteriors  
Standard shade : XBW (Extra Bleaching White), BW (Bleaching White), A1, A2, A3, A3.5, A4, B1, B2, B3, C3, CV (Cervical), CVD (Cervical Dark).

Outside special shade : WT (White Translucent), DT (Dark Translucent), CT (Clear Translucent), NT (Natural Translucent), GT (Gray Translucent), CVT (Cervical Translucent).

2.6 shades for posteriors  
Standard shade : P-A1, P-A2, P-A3, P-A3.5, P-A4  
Outside special shade : P-WT (White Translucent), P-NT (Natural Translucent).

Note:

- A, B, C, AO shades are based on Vita® Shade.
- \*Vita is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

**STORAGE**

Store in a cool and dark place (4 – 25°C / 39.2 – 77.0°F) away from high light exposure or direct sunlight.

(Shelf life: GRADIA DIRECT ANTERIOR - 3 years from date of manufacture / GRADIA DIRECT POSTERIOR - 3 years from date of manufacture)

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**CAUTION**

- 1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water.
- 2. In case of contact with eyes, flush immediately with water and seek medical attention.
- 3. Take care to avoid ingestion of the material.
- 4. Wear plastic or rubber gloves during operation to avoid direct contact with any inhibited resin layers in order to prevent possible sensitivity.
- 5. For infection control reasons, Unitips are for single use only.
- 6. Wear protective eyewear during light curing.
- 7. When polishing the polymerised material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- 8. Do not mix with other similar products.
- 9. Avoid getting material on clothing.
- 10. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.
- 11. Do not use GRADIA DIRECT in combination with eugenol containing materials as eugenol may hinder GRADIA DIRECT from setting.

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