


**Faster.
Easier.
Smarter.**

Access Crown® sets to an elastic memory state in only 60 seconds.

Easy to remove without distortion.

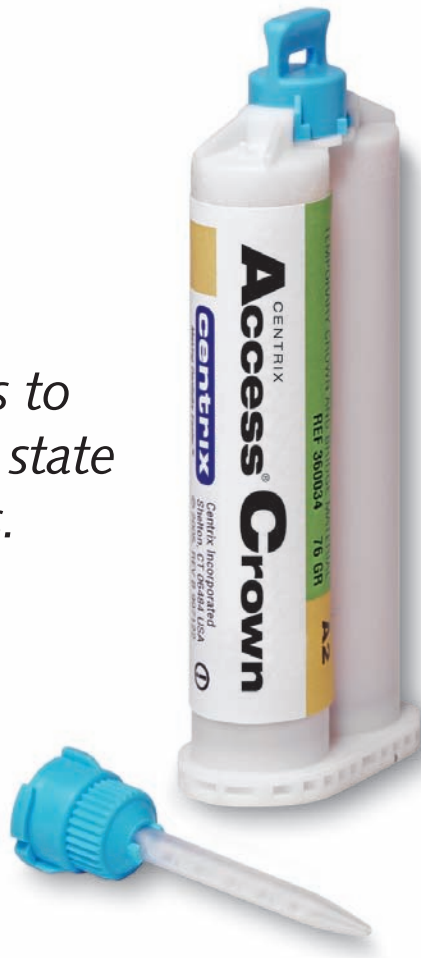

Only AccessCrown delivers the most shades and the fastest setting time on the market.

No temporary bis-acryl composite is faster or easier to use! Sets to an elastic-like state, ready for removal from the mouth in just one minute. The primary clinical advantage of this bis-acryl composite is its elastic behavior during polymerization. This memory will allow you to pull it from undercuts or non-parallel tooth preparations and return to its original form.

- Ideal handling characteristics — sets in the mouth in 60 seconds
- Low shrinkage, low heat, high compressive strength
- Enhanced color stability, less staining and radiopaque
- Seven popular VITA* shades, plus bleach shade
- 40% less waste due to smallest possible mixing tips
- MMA-free: no unpleasant taste or smell
- Can be easily repaired with flowable composite
- Compatibility with both eugenol and non-eugenol cements
- High polishability
- A 1:1 ratio automix cartridge system (same as your impression gun)

* Not a trademark of Centrix, Inc.

Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit centrixdental.com for more information.



CENTRIX
AccessCrown®

Bis-Acryl Composite for Provisional Crowns and Bridges




centrix®

Making Dentistry Easier.™

INSTRUCTIONS OF USE:



1) Prep tooth as normal.

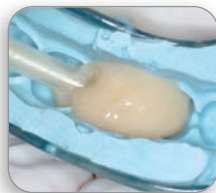

2) Just inject into pre-operative impression, insert into mouth and wait one minute.



3) Access Crown turns semi-elastic for easy removal.





4) Trim with composite instruments, repair with flowable composite.



5) The final self-cure set takes only 2.5 minutes more.

COMPARE:

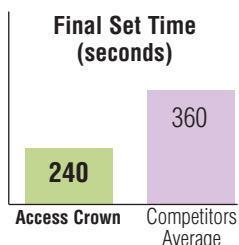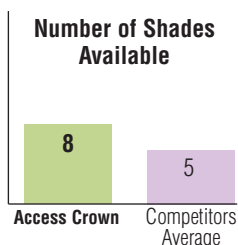

ORDERING INFORMATION:

ACCESS CROWN

Kit contains: One 76-gram cartridge plus 10 automix nozzles
Available in the following shades:

- REF 360029 A3.5
- REF 360030 A3
- REF 360031 B1
- REF 360032 B3
- REF 360034 A2
- REF 360035 A1
- REF 360037 C2
- REF 360038 BL Bleach shade



SUGGESTED ACCESSORIES:

- REF 360036 DS50 1:1 Automix Syringe
- REF 910626 DS50 Crown nozzles, 50 (blue hub)

RISK FREE Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Ordering is easy.

Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com



centrix®

Making Dentistry Easier.™