

Patterson®

Dental Aspirating Syringes

Directions for Use

Rx ONLY

A Type - 089-9773 CW Type - 089-9765

Description

Patterson® Aspirating Syringes are used in conjunction with a needle and a 1.8cc pre-filled anesthetic cartridge (not supplied by Patterson®). Aspirating syringes are available in Type CW (Cook-Waite) and Type A (Astra). The syringes are reusable and are provided clean and supplied non-sterile.

Indications

Patterson® Aspirating Syringes are indicated to be used in conjunction with anesthetic needles and anesthetic cartridges for injection of anesthetic solutions in the oral cavity.

Contraindications

None known.

Warnings

- To avoid needle sticks, re-cap needle after injection has been given.
- To avoid leakage during an injection, ensure that needle is straight as it penetrates the cartridge diaphragm.

Precautions

- Syringes must be sterilized prior to use. See sterilization instructions.
- Hubs cannot be removed. Do not attempt to remove hub from syringe for sterilization or for any reason as damage to the syringe will occur.
- Do not mix metals when sterilizing syringes. Sterilizing syringes with other non chrome-plated brass instruments may cause damage to syringes.
- Do not use cold sterilants, as they will react to the chrome plating of the syringe.
- Keep needle capped while affixing or removing from syringe.

Instructions

Loading Instructions

1. Retract piston rod fully.
2. Insert 1.8cc cartridge, plunger end first, into syringe barrel.
3. Hold cartridge firmly in place. Push piston rod forward with a quick jab.
4. Affix needle to aspirating syringe.
5. Expel a few drops of solution to clear needle.
6. Aspirate before injecting solution after inserting needle into tissue.
7. After positioning the needle, pull back on thumb ring to aspirate.

Unloading Instructions

1. Remove needle from syringe.
2. Withdraw piston rod to clear cartridge for removal.
3. Tilt syringe to expel cartridge.

Cleaning Instructions

- Remove all traces of blood, saliva, and anesthesia solution.
- Remove paraffin that has accumulated on the harpoon.

Sterilization Instructions

- Syringes are supplied non-sterile and must be sterilized prior to each use.
- Syringes should not be sterilized with other instruments to avoid syringe damage.
- Before each use, sterilize and inspect instrument. Sterilize syringes according to an approved National/International Standard. Patterson® syringes have been tested with no adverse effects under the following sterilization parameters in a standard, gravity steam, sterilizer:
 - Temperature: 250°F (121°C)
 - Exposure Time: 30 minutes
 - Dry Time: 20 minutes
- Do not use cold sterilants, as they will react to the chrome plating of the syringe.

Manufactured for:

Fabrique' pour:


PATTERSON
DENTAL
Patterson Dental Supply, Inc.
1031 Mendota Heights Road
Saint Paul, MN 55120
08997DFU Rev. A 09/11