

RapidCore's depth of cure just got faster and more efficient, with less waste.



Actual size. Just snap the cartridge in place.



Single-dose dispensing provides for the fastest core buildup, without a heavy, clumsy automix syringe!

It's even easier. Simply snap RapidCore® MiniMix® cartridge into a C-R® Syringe and squeeze the material through the exclusive mixing nozzle needle tip. You're ready to cut the prep in two minutes. RapidCore's advanced thixotropic formula flows smoothly through the needle yet will not slump, making core forms and matrix bands unnecessary.

- Two-minute set time reduces chair time by 50%
- Pure self cure — no depth of cure concerns
- Needle tip allows delivery of material to bottom of a post hole
- Non-slumping formula so there is no need to use a matrix
- Cuts like dentin to reduce ditching
- MiniMix technology virtually eliminates waste
- Available in three shades – natural, contrast and white
- Radiopaque

| | |
|----------------------------|--|
| Bond strength: | 21 Mpa with Adhere™ Bonding System |
| Working time (Room temp.): | 60 seconds |
| Set time in mouth: | 2 minutes |
| Comprehensive strength: | 248 Mpa |
| Water Extractable: | Less than 0.02 mg/cm ² 7 days @ 37°C |

Ordering is easy. Call 800-235-5862 or contact your preferred dealer. Visit centrixdental.com for more information.



NEW! .5ml single-dose MiniMix cartridge

CENTRIX
RapidCore®
MiniMix®

Self-Cure Composite Resin
Core Buildup Material



centrix®
Making Dentistry Easier.™

Self-Cure Composite Resin
Core Buildup Material

**6-STEP GUIDE TO BUILDING A VOID-FREE,
BONDED, COMPOSITE POST & CORE**



1. Enlarge root canal to accept RapidCore needletip. Bevel the edges to eliminate sharp angles.



2. Etch the post hole, rinse thoroughly and dry.



3. Apply bonding agent according to manufacturer's instructions.



4. Insert needletip to the bottom of the post hole and inject material with slow steady pressure.



5. Continue filling by injecting and slowly withdrawing the needletip.



6. Allow material to fully cure and then cut and shape the core. The result is a solid, securely bonded post and core.

ORDERING INFORMATION:

RapidCore MiniMix Kits (12) include:
12 x single-dose cartridges (0.5ml net ea)
12 mixing nozzles

- REF 310126 Natural shade
- REF 310127 White shade
- REF 310128 Contrast shade



RapidCore MiniMix Kits (36) include:
36 x single-dose cartridges (0.5ml net ea)
36 mixing nozzles

- REF 310256 Natural shade
- REF 310257 White shade
- REF 310258 Contrast shade



Our new Snub Nose syringe is ideal to use with our MiniMix cartridges. Just "snap" into place without the aid of a plunger, and once material is delivered, empty cartridges are "snapped" out.

SUGGESTED ACCESSORY

- REF 130651 Snub Nose Syringe

RapidCore 50ml Kits include:
1 50ml cartridge, 15 mixing nozzles
and 15 dispensing tips

- REF 310133 White shade
- REF 310134 Natural shade



RapidCore 25ml Kits include:
1 25ml cartridge, 10 mixing nozzles
and 10 dispensing tips

- REF 310136 White shade
- REF 310137 Natural shade
- REF 310138 Contrast shade



SUGGESTED ACCESSORIES

- REF 310251 Adhere™ Adhesive
- REF 310252 Adhere Activator

RISK FREE Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.

Product covered by one or more of the following U.S. patents: 5,052,927; 5,336,088; and D359,119. Other U.S. and International patents pending or granted.
© 2009 Centrix, Inc. All rights reserved. RapidCore and MiniMix are registered trademarks of Centrix, Inc. Adhere is trademarks of Centrix, Inc.

Ordering is easy.
Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com



centrix®
Making Dentistry Easier.™