




Unprocessed state of a fully anatomical resin nano-ceramic crown

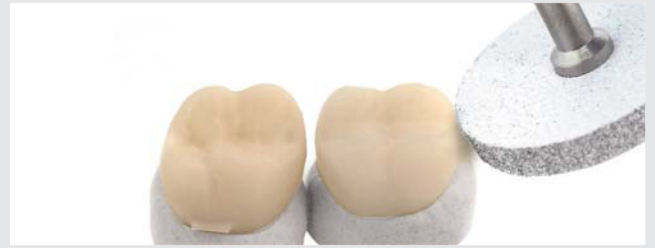

9511U: Trimming the contact point,
Recommended rotary speed: 8.000 - 12.000 rpm



9507P: Smoothing of the occlusal surface and crown external shape, Recommended rotary speed: 7.000 - 12.000 rpm



9507H: High gloss polishing of the occlusal surface and crown external shape, Recommended rotary speed: 7.000 - 12.000 rpm




9508H: Targeted high gloss polishing, e.g. of fissures,
Recommended rotary speed: 7.000 - 12.000 rpm



100: Working the polishing paste into the material surface using a soft polishing brush, Recom. rotary speed: 10.000 - 15.000 rpm



150: High gloss polishing of the crown with a cotton buff,
Recommended rotary speed: 7.000 - 12.000 rpm

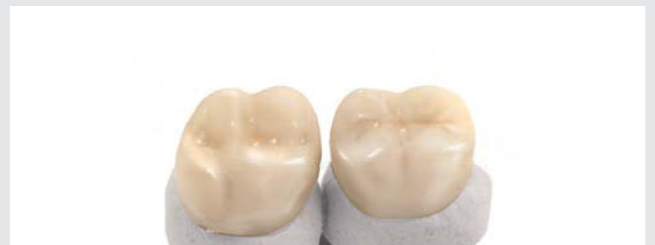

Result: Excellent high gloss by polishing with LUSTER® set 3M000