Patterson® Disposable Three Way Impression Trays

Directions for Use:

1. After completion of tooth preparations and gingival retraction, place unloaded tray. When using the posterior, quadrant or sideless, make sure the crossbar of the tray is distal to the last maxillary tooth and the tray is centered buccally/lingually.
2. Have the patient "close" on the unloaded tray and check the opposite side of the patient's mouth to make sure that the patient has closed into the correct occlusion.
3. Paint impression adhesive on tray walls only, not on webbing. Overload both sides. Note: Must be used with a firm-setting material. It also may be used with hard-setting Heavy Body/Light Body one-step techniques, not two-step techniques.
4. Remove retraction cord, if used. Syringe impression material around important areas.
5. With syringed impression material in place and still unset, place loaded tray into proper position.
6. Have patient close properly.
7. When set, gently remove from mouth. Inspect final impression. No plastic pressure areas should be visible.
8. Disinfect.

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Patterson Dental Supply, Inc.