

←  
Fill here for  
three operatories  
Mix 1½ oz.  
cleaning solution  
with 3 qt. water.  
Begin with  
Operator 1  
nozzles.

←  
Fill here for  
two operatories  
Mix 1 oz. cleaning  
solution with  
2 qt. water.  
Begin with  
Operator 2  
nozzles.

←  
Fill here for  
one operator  
Mix ½ oz. cleaning  
solution with  
1 qt. water.  
Begin with  
Operator 3 nozzles.

For use with Biotrol's dental evacuation system cleaners:



**VACUSOL™ ULTRA & NeutraVAC™**

# BIOTROL EZ 1-2-3™ ATOMIZER

Clean up to **THREE** operatories with **ONE** mix

#### INITIATE USE:

1. Remove lid. Dispense ½ ounce of Biotrol evacuation cleaning solution into the EZ 1-2-3™ Atomizer per each set of operator lines to be cleaned. The EZ 1-2-3™ Atomizer cleans a maximum of three sets of operator lines in one mix.
2. Add one quart of water into the EZ 1-2-3™ Atomizer per ½ ounce of Biotrol evacuation cleaning solution.
3. Replace lid and gently shake Atomizer to mix solution.

#### CLEANING:

1. Attach saliva ejector adapter to small diameter nozzle and attach one or two high speed volume adapters (as needed) to remaining nozzles for corresponding operator.
2. When cleaning 3 operatories in one mix, begin with Operator 1 nozzles. When cleaning only 2 operatories in one mix, begin with Operator 2 nozzles. When cleaning only 1 operator in one mix, use only Operator 3 nozzles.
3. With vacuum pump on and handpiece valves open, aspirate the diluted cleaning solution through the evacuation tubing.

#### FINISH AND REPEAT:

1. Disconnect the tubings and return to their usual location in the operator.
2. Repeat "Cleaning" steps for up to two additional operatories in one EZ 1-2-3™ mix.
3. The EZ 1-2-3™ Atomizer is not autoclavable. Rinse EZ 1-2-3™ Atomizer with water when finished and allow to air-dry.

**BIOTROL RECOMMENDS CLEANING EVACUATION LINES DAILY FOR OPTIMUM PERFORMANCE OF YOUR EVACUATION PUMP**

REORDER NO. ED592



# BIOTROL

13705 Shoreline Court East, Earth City, MO 63045  
800-822-8550 • [www.biotrol.com](http://www.biotrol.com)

12002389A

For use with Biotrol's dental evacuation system cleaners:



**VACUSOL™ ULTRA & NeutraVAC™**