




## PRR & SMALL CLASS I RESTORATION

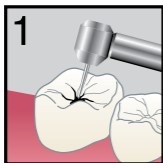


1  
Prepare tooth as needed,  
rinse, leave damp.

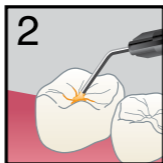


2  
Apply **BEAUTIFIL kids SA**  
in a thin layer ( $\leq 0.5\text{mm}$ )  
with needle tip or hand  
instrument.  
Leave for 20 sec.

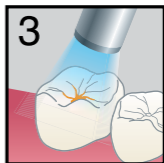


3  
Light cure for 5 sec (LED).

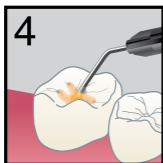

4  
Apply additional  
increments ( $\leq 2\text{mm}$ ) as  
needed.

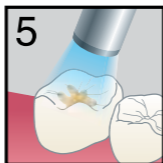

5  
Light cure each increment  
for 10 sec (LED).

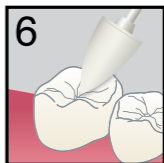

6  
Finish and polish with  
OneGloss PS or another  
polisher.

Visit [shofu.com](http://shofu.com) Call 800.827.4638

Shofu Dental Corporation San Marcos, CA