



5% Sodium Fluoride White Varnish

Each mL of varnish contains 50 mg of Sodium Fluoride (equivalent to 22.6 mg fluoride ions).

Indications For Use

Braval® 5% Sodium Fluoride White Varnish is a fluoride containing preparation for use as a cavity varnish and for the treatment of dentin hypersensitivity.

Contraindications:

1. Known allergic reactions to fluoride.
2. Ulcerative gingivitis and stomatitis.
3. Avoid ingestion during application.

Precautionary Information for Dental Personnel

Use only as directed. To limit ingestion, determine the proper dosage amount for the patient. Discontinue patient's use of other prescriptive fluoride preparations (i.e. gels, rinses) for 24 hours. Patients taking fluoride supplements should discontinue use of supplements for 2-3 days following treatment.

Warning

Contains ingredients that may be irritating to the skin and eyes. Edematous swelling has been reported only in rare instances, especially following application to extensive surfaces with fluoride varnish. Dyspnea, although extremely rare, has occurred in asthmatic people.

Instructions for Use

1. Clean teeth and remove obvious plaque and calculus from area to be treated. For best results, remove any excess moisture from the tooth surface.
2. Peel open the foil lidding material.
3. Using the brush provided, mix varnish until it reaches an even consistency.
4. Paint a thin layer of varnish onto treatment area using broad and even strokes.
5. Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth.
6. When dry, any excess varnish may appear "spotty" and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth.
7. Dispose package and applicator after use.
8. Patient may leave immediately after treatment. Patient should be advised to not brush teeth for a minimum of 2 to 4 hours after treatment.

For professional use only



Single Use Only



Consult accompanying documents

Item #	Packaging	Flavors
133-3699	200/box	Mint
133-3855	200/box	Melon



28°C
82°F
max
Store at a controlled
room temperature
15°C
59°F
min

Manufactured for:
Patterson Dental Supply, Inc.
1031 Mendota Heights Road
Saint Paul, MN 55120

Made in USA

296626A Rev.00