

G4571/0817



MADE IN U.S.A. – Fabriqué aux É.-U.

Manufactured for:
Fabriqué pour:
Patterson Dental Supply, Inc.
1031 Mendota Heights Road
Saint Paul, MN 55120

Open this end
Ouvrir de ce côté

Contains one quality instrument *Contient un instrument de qualité supérieure*

Intended for use only by dental professionals. Inspect, clean and sterilize before each use. Steam sterilize for at least 4 minutes at 270°F/132°C or 30 minutes at 250°F/121°C. Recommend 30 minutes dry time after sterilization cycle. Do not heat above 350°F/177°C. Use only recommended chemicals. Do not disinfect with phenols.

Conçu uniquement pour une utilisation par un professionnel dentaire. Inspecter, nettoyer et stériliser avant chaque utilisation. Stériliser à la vapeur durant au moins 4 minutes à 132 °C (270 °F) ou durant 30 minutes à 121 °C (250 °F). Temps de séchage recommandé de 30 minutes suivant le cycle de stérilisation. Ne pas exposer à des températures supérieures à 177 °C (350 °F). N'utiliser que les produits chimiques recommandés. Ne pas désinfecter à l'aide de produits à base de phénols.