# Patterson® In-Office Whitening Kit

## Description:

Patterson's In-Office 35% is a hydrogen peroxide, in-office whitening gel. The 2-barrel jet mix syringe ensures maximum strength for each application and precise dosing of the activator. The final hydrogen peroxide concentration is 35%.

## Indications for Use:

Patterson's In-Office 35% whitening is for in-office use only. It is applied by the dental professional for bleaching one or more teeth. Patterson's In-Office Whitening is also used on non-vital teeth, including in-office intracoronal bleaching.

Patterson's In-Office Whitening is an alternative, conservative method (compared to crowns, veneers, etc.) for bleaching dark, internally discolored teeth caused by predisposing factors such as disease, traumatic or iatrogenic injury, congenital, systemic, metabolic or pharmacological influences. These can include but are not limited to elevated bilirubin levels, tetracycline and adult minocycline stains, porphyria, erythroblastosis fetalis and high fluoride intake during tooth development.

## Preliminary Procedures:

1. Clean and polish teeth approximately two weeks prior to bleaching appointment.
2. Determine and record baseline shade. Use photographs when indicated for baseline documentation.
3. Prior to mixing, bring Patterson's In-Office Whitening to room temperature.
4. **WARNING:** Patient, clinician and assistant must wear protective eyewear and gloves when mixing and applying this product.

## Directions for Use:

### Mixing Patterson's In-Office Whitening (Fig.1):

1. Check syringes are securely attached.
2. Press the plunger of the Whitening syringe in, pushing all contents into the Activator syringe.
3. Press the plunger completely back into the syringe.
4. To thoroughly mix the Activator with the Whitening Gel, push stems back and forth continually with thumbs and mix a minimum of 50 times rapidly (25 times each side).
5. Press all mixed gel into the Whitening Gel syringe.
6. Twist to separate the two syringes and securely attach the recommended tip (Micro 20ga tip) onto the Whitening Gel syringe.

### Isolation - Gingival Barrier (recommended isolation):

1. Place bite block and self-supporting lip/cheek retractors. (Fig.2)
2. Remove Luer lock cap from Gingival Barrier resin barrier syringe and securely attach a Micro 20ga tip.
3. Verify flow prior to applying intraorally.
4. Rinse and dry teeth and gums thoroughly.
5. Express a continuous bead of Gingival Barrier resin along the gingival margin, overlapping approximately 0.5mm onto the enamel. Begin and finish the bead one tooth beyond the most distal tooth that is being bleached.
6. Continue building the barrier apically until it is 4-6mm high and 1.5-2.0mm thick. (Fig.3)
7. Express resin through any open embrasures onto the lingual tissue to completely seal and cover exposed papilla.
8. Using the mouth mirror, look up the long axis of the teeth for gaps or minute openings exposing the underlying tissue and re-apply resin.
9. Light cure 20 seconds per arch using a scanning motion. Carefully check resin cure with an instrument. (Fig.4)

### Isolation - Rubber Dam:

1. Apply rubber dam placing ligated clamps bi-laterally and one tooth distally to the teeth to be whitened.
2. Before attaching rubber dam to the frame, apply a bead of rubber dam sealer at the gingival margin beneath the dam.
3. Attach dam to the frame and invert rubber dam at the gingival border. Check for optimum seal.

### Applying Patterson's In-Office Whitening:

1. On gauze or mixing pad, verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips.
- NOTE: If resistance continues, DO NOT use that syringe and contact Patterson Customer Service for replacement.
2. Apply a 0.5-1.0mm thick layer of gel to the clean labial surface of the tooth.
3. Allow gel to remain on the teeth 20 minutes, periodically checking and re-applying areas that have thinned or need replenishing.
4. Suction gel from teeth using a surgical suction tip. To avoid gel splatter, do not use water while suctioning gel (Fig.6). When no gel is visible, lightly rinse and air dry. Use caution not to dislodge the isolation barrier or rubber dam seal.
5. Evaluate shade change after each application and monitor patient for sensitivity and irritation. If persistent tooth sensitivity occurs, discontinue use.
6. Avoid contact with soft tissue or salivary flow. If chemical irritation occurs due to the intraoral soft tissue coming in contact with the hydrogen peroxide gel, immediately wipe the gel from the tissue using a wet gauze and rinse the area well. Tissue should return to normal within 20-30 minutes.
7. Repeat steps 2-5 up to two additional times if desired results have not been achieved. Do not exceed 3 applications per visit.

8. If additional in-office whitening is desired and no significant sensitivity is noted, re-schedule and retreat patient as desired. If sensitivity was experienced, repeat treatment when the sensitivity has subsided.

## Clean-up:

1. Suction gel from teeth using a surgical suction tip, then thoroughly rinse teeth with an air/water spray and the high-volume suction. (Fig.7)
2. Using a dental instrument, gently slide the tip beneath the Gingival Barrier and lift off. Check and remove any interproximal remnants.
3. If using a rubber dam, follow "Clean-up" Step 1, remove clamps, dam, and frame.
4. Remove rubber dam sealer with spatula, soft toothbrush, floss, suction and firm air/water spray.

## NOTES:

- Whitening is used to return shade of natural teeth to the shade of the remaining non-discolored teeth or previously matching restoration.
- Whitening effect may continue for 24-48 hours following bleaching, hence a slight increase in whitening may be realized during this period.
- A single, or a few teeth can be bleached using this same instruction and applying the bleach only to those specific areas of discoloration.
- For post-op sensitivity, use potassium nitrate products such as a desensitizing gel immediately after treatment and for 15 minutes to 1 hour per day until sensitivity subsides.
- Additional whitening after successful treatment of a single tooth or some teeth is possible by either Patterson In-Office Whitening Gel or a take-home system as soon as any sensitivity has dissipated.
- Etching enamel or prophy paste polishing before any bleaching procedure is not recommended and does not potentiate the bleaching process, but can cause sensitivity.

## Endodontically Treated Teeth:

- Non-vital teeth can be bleached from both the inside out and the outside in to accelerate whitening speed/effectivity.
1. Completely remove all the restorative and root sealing material from the coronal pulp chamber and 2-3mm below the gingiva. Place a 2mm thick conventional glass ionomer or a resin modified glass ionomer to seal the canal.
  2. Follow the "Mixing" instructions 1-6 above.
  3. Attach the Micro 20ga tip. Verify flow prior to applying intraorally.
  4. Follow "Isolation" instructions above isolating both the labial and lingual sides of the tooth. (Fig 8)
  5. Express into access opening, the labial, and if desired, the lingual surface. (Fig 9 and 10)
  6. Allow gel to remain in the tooth 20 minutes.
  7. Suction gel from access opening and tooth using a surgical suction tip. To avoid gel splatter, do not use water while suctioning gel.
  8. When no gel is visible, lightly rinse and air dry. Use caution not to dislodge the isolation barrier or rubber dam seal.
  9. Evaluate shade change.
  10. Repeat steps 5-9 as needed.
  11. Follow "Clean-up" instructions to remove gingival barriers.
  12. Seal access with a non-eugenol and non-resin provisional restorative.

## NOTES:

- If additional whitening is desired, re-schedule and repeat the Patterson's In-Office Whitening procedure above.
- Whitening effect may continue for 24-48 hours following bleaching, hence a slight increase in whitening may be realized during this period.
- Over years, regression is to be expected and re-bleaching on an average of 3-5 years is recommended.

## Precautions and Warnings:

1. Gingival and oral health should be confirmed before using this product.
2. Clinician, assistant, and patient must wear protective eyewear with side shields when mixing and applying Patterson's In-Office Whitening.
3. Restorative materials will not whiten and therefore it is recommended that teeth are whitened prior to placing esthetic restorations.
4. Wait 7-10 days after bleaching procedures before placing adhesive restorations.
5. Carefully read SDS and all instructions before using.
6. Bring syringes to room temperature before mixing.
7. Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. If resistance continues, DO NOT use that syringe and contact Patterson Customer Service for replacement.
8. Use only recommended tips. Do not reuse tips to avoid cross contamination.
9. To avoid cross-contamination, use a disposable syringe cover, re-cap, and wipe syringe with an intermediate disinfectant between uses. If these measures are not taken, the syringes should be considered single-use.
10. Keep out of reach of children.
11. Pregnant or breastfeeding women, or patients with serious health concerns should not whiten. Consult their primary care provider prior to treatment.
12. Wear protective gloves when handling this product.
13. Before disposing of syringes aspirate water into the syringe and express liquid down the drain. Repeat a couple of times before disposing of the syringe. Make sure any gauzes used are rinsed with water.
14. Do not use on patients with a known sensitivity to peroxides, glycols, acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
15. When protecting a full arch, light cure the resin barrier on standard mode using a scanning motion for 20 seconds and keeping the lens

- approximately 1 cm from the barrier and tissue. When protecting tissue adjacent to fewer teeth, scan the barrier for approximately 3-5 seconds per tooth. With a dental instrument, carefully check resin for a full cure.
16. If bubbles occur along the margin of the Gingival Barrier after the application of Patterson's In-Office Whitening gel, suction off gel, rinse and check for leakage. Dry area and re-apply Gingival Barrier to the site.
17. Cover any sensitive areas of exposed dentin.
18. For patients with known tooth sensitivity, pre-op and post-op treatment with desensitizing gel, sensitivity relief toothpaste, and/or fluoride varnish is recommended. Depending on the severity of sensitivity, other treatment options are available.
19. Areas of hypocalcification, though not clinically visible, may exist and will whiten quicker than surrounding enamel. As patient continues to whiten, the areas will begin to blend.
20. All Patterson syringe-delivered dental materials should never be injected.
21. Keep out of direct heat and/or sunlight.
22. Dispose of tips and syringe as described in above instruction and according to local regulations.

# Patterson's® Gingival Barrier

## Product Description

Patterson's Gingival Barrier is a unique, light-reflective, passively adhesive (sealing) light cured resin. Its methacrylate base is designed with adequate strength to maintain a barrier, yet removes easily from embrasures and undercuts. The contrasting color of Patterson's Gingival Barrier shows its location on the gingiva and teeth for more effective coverage. For all products described, carefully read and understand all instructions and SDS information prior to use.

## Delivery Form(s)

1.2mL syringe  
Micro 20 ga Tip

## Indications for Use/Intended Purpose

Patterson's Gingival Barrier is a unique, light-reflective, passively adhesive (sealing) light cured resin. Its methacrylate base is designed with adequate strength to maintain a barrier, yet removes easily from embrasures and undercuts.

## Contraindications

Do not use Patterson's Gingival Barrier on patients with known sensitivity to resins, especially methacrylates. If dermatitis, rash or any other allergic reaction develops, wash area with lots of water and refer patient their physician.

## Warnings and Precautions

Resins can be sensitizing. If bubbles occur at the gingival margin of the Gingival Barrier, reapply in that area.

To avoid cross-contamination, wipe syringe with an intermediate level disinfectant between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single-use.

Tips are single-use only.

When curing resins on soft tissues, use scanning motion to prevent heat irritation from curing light.

## Stepwise Instructions

### Preparation

Remove luer lock cap and attach recommended tip securely onto the Gingival Barrier syringe.

Verify flow prior to expressing intraorally. If resistance is met, replace tip and re-check.

Completely rinse and dry teeth and gingiva where Gingival Barrier will be applied.

### Use

Express Patterson's Gingival Barrier resin approximately 4-6mm in height and lap approximately 0.5mm onto enamel. Extend resin beyond the last tooth to be treated and onto open embrasures, to protect interdental papilla and lingual gingival tissue. Resin buttons extending through gingival embrasures add retention to the Gingival Barrier.

When protecting a full arch, light cure the resin barrier on standard mode using a scanning motion for 20 seconds and keeping the lens approximately 1cm from the barrier and tissue. With a dental instrument, carefully check resin for a full cure. If not using on full arch, scan the barrier for approximately 3-5 seconds/tooth.

### Cleanup

After treatment, lift the Gingival Barrier from surface by gently sliding a dental instrument between the dam and tissue. All or large pieces will lift off at one time.

Remove the Gingival Barrier interproximally using explorer and/or floss. Rinse mouth thoroughly.

### Processing

Wipe down syringes with an intermediate level disinfectant between uses. If syringe barrier sleeves are not used they are considered single-use.

### Storage and Disposal

Dispose of waste according to local rules, guidelines, and regulations.

### Miscellaneous Information

For product SDS, reordering, and/or complete description of Patterson's product line, please visit our website [www.pattersondental.com](http://www.pattersondental.com) Report any serious incident to the manufacturer and the competent authority.

## Directions for use:

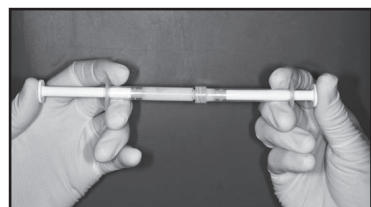
Fig. 1 Refer to the "Mixing Patterson's In-Office Whitening" section for mixing instructions.

Fig. 2 Remove any plaque or debris with brush then place bite block.

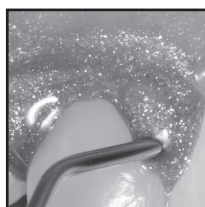
Fig. 3 Express Gingival Barrier to build a 4-6mm by 1.5-2mm thick strip onto gingiva. Lap approximately 0.5mm onto enamel.

Fig. 4 Cure resin with a quality curing light by scanning the area for 20 seconds.

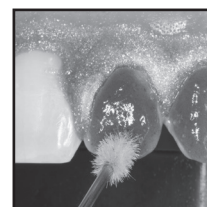
Fig. 5 Apply the Whitening Gel, about 0.5-1.0mm thick, onto labial surfaces. Extend slightly onto incisal/occlusal edge.

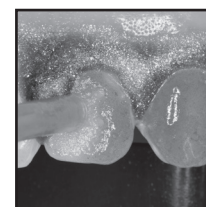
Fig. 6 Remove Whitening Gel using a surgical suction tip.

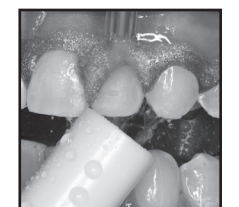
Fig. 7 Rinse thoroughly directing water toward high-volume suction.

## Endodontically Treated Teeth

Fig. 8 Isolate the lingual and labial gingiva and papilla.

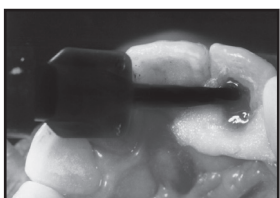
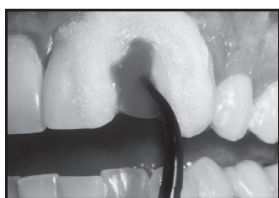
Fig. 9 and Fig. 10 After the GI plug is fully set, express Whitening Gel into the access opening and to the labial and lingual surfaces.

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Patterson Dental Supply, Inc.