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
EN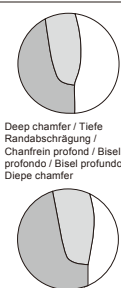

GC Initial™ LiSi Block

LITHIUM-DISILICATE GLASS CERAMIC CAD/CAM BLOCK

For use only by a dental professional in the indications.

INDICATIONS FOR USE

- Veneers
- Crowns
- Inlays
- Onlays
- Partial crowns
- Crown or splinted crown on top of an implant abutment

PREPARATION DESIGN

When designing restorations, the following preparation guidelines and minimum dimensions should be maintained.

Preparation guidelines for all-ceramic restorations

- Prepare margins with deep chamfer or rounded shoulder. (Fig. 1)
- Avoid having margins in direct occlusal contact with the opposing tooth.
- Antagonist contacts must be taken into consideration.
- The incisal edge of the preparation should be at least 1,0 mm (milling tool geometry) in order to permit optimum milling during CAD / CAM fabrication.

Fig. 1 Abb. 1

Veneers (Fig. 2)

- If possible, the preparation should be located in the enamel.
- Reduction in the cervical and / or labial area by 0,6 mm, and the incisal edge by 0,7 mm.

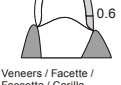
Fig. 2 Abb. 2

Anterior crown (Fig. 3)

- Width of the shoulder/chamfer at least 1,0 mm.
- Reduce the incisal edge by approx. 1,5 mm.
- Reduce the vestibular and / or oral area by approx. 1,2 mm.

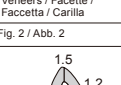
Fig. 3 Abb. 3

Inlay (Fig. 4)

- A preparation depth of at least 1,0 mm and an isthmus width of at least 1,0 mm must be observed in the fissure area.
- Do not prepare undercuts.
- Ensure that the cavity walls form an angle of 6° with the long axis of the tooth.
- All internal edges and angles should be rounded.

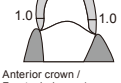
Fig. 4 Abb. 4

Onlay (Fig. 5)

- A preparation depth of at least 1,0 mm and an isthmus width of at least 1,0 mm must be observed in the fissure area.
- Do not prepare undercuts.
- Ensure that the cavity walls form an angle of 6° with the long axis of the tooth.
- All internal edges and angles should be rounded.
- Provide at least 1,0 mm of reduction in the cusp areas.

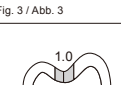
Fig. 5 Abb. 5

Partial crown

- Provide at least 1,5 mm of reduction in the cusp area.
- Width of the shoulder / chamfer should be at least 1,0 mm.

Fig. 6 Abb. 6

Posterior crown (Fig. 6)

- Reduce the anatomical shape and observe the stipulated minimum thickness.
- Width of the shoulder / chamfer should be at least 1,0 mm.
- Reduce the incisal crown third – in the occlusal area by approx. 1,5 mm.

CAD/CAM PROCESSING

Please refer to respective instructions of the milling unit.

Note:

Blocks can be processed with authorized CAD/CAM systems.

FINISHING AND POLISHING

Remove the sprue and try the final restoration. When necessary, adjust proximal and occlusal contact with the opposing tooth. Initial LiSi Block can be either milled and polished or milled, glazed and fired.

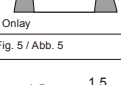
Fig. 7 Abb. 7

Polishing

Finish and polish using suitable grinding instrument, low rpm and light pressure.

Pay attention to margins and contact points when polishing the restoration.

Overheating of the glass-ceramic must be avoided.

Clean the restoration before cementing.

GLAZE AND STAINING (Option)

For glazing and staining, please use our dedicated GC Initial Lustre Pastes NF/ONE and GC Initial Spectrum Stains for best results.

• Please follow the manufacturers' instructions for use of the materials of glazing and staining.

• When firing, do not heat or cool the restorations quickly. Rapid change in temperature could break the material. When firing, proper furnace tray (honeycomb tray) and support pin (in combination with the GC Initial Firing Foam) should be used, contact information is on page 10.

CEMENTATION

1. Restoration Etching protocol

- Apply hydrofluoric acid gel (5-9%) for 20 seconds to the inner surfaces of the restoration.
- Wash with water spray or an ultrasonic cleaner and dry.
- Condition the etched surfaces with a silane coupling agent (such as CERAMIC PRIMER II or G-Multi PRIMER) and allow it to dry.
 - Hydrofluoric acid is for extraoral use only and must not be applied in the oral cavity.
 - Phosphoric acid (35-37%) can be also used for the purpose of cleaning the surface, preferably scrubbing with a microbrush for 10-15 seconds.

2. Cementing

- Cement with an adhesive resin cement (such as G-CEM LinkForce) or a self-adhesive resin cement (such as G-CEM ONE).
 - Prilož to using CERAMIC PRIMER II, G-Multi PRIMER, G-CEM LinkForce or G-CEM ONE, refer to the respective instructions for use.
 - In case the preparation is non-retentive, an adhesive resin cement (such as G-CEM LinkForce) is preferred.

SHADE

HT: (High Translucency) A1, A2, A3, B1
LT: (Low Translucency) A1, A2, A3, B1

SIZE

14

PACKAGE

5 blocks per box

STORAGE

Recommended for optimal performance, store at room temperature (4-25°C/39-77,0°F) away from direct sunlight and high humidity.

CAUTION

- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Not suitable for patients with:
 - Severely reduced residual dentition
 - Severely reduced occlusal vertical dimension
 - Parafuncions (e.g. suffering from bruxism)

- Restorations should not be blasted with alumina or glass beads.
- Do not use non-designated material.
- For use only as indicated.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

or for The Americas

<http://www.gcamerica.com>

They can also be obtained from your supplier.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

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