Patterson[®] Desensitizer

CAUTION

DO NOT USE
IF PACKAGE IS
DAMAGED

READ
INSTRUCTIONS

TEMP.
RANGE FOR
STORAGE

PRESCRIPTION
REQUIRED

KEEP AWAY
FROM LIGHT/
HEAT

HEALTH
HAZARD

IRRITANT
(SKIN &
EYES)

CORROSIVE

EN

DESCRIPTION:

Patterson[®] Desensitizer is a powerful desensitizer that acts to prevent bond degradation, rewets the tooth surface prior to bonding, and optimizes the tooth / restoration interface. Bonding protocols often call for a “moist” or “shiny” or “damp” surface for bonding. Patterson[®] Desensitizer eliminates the guess work by rewetting to the proper level every time. Finally, Patterson[®] Desensitizer provides fluoride and a proprietary ingredient to the surface of the tooth to create a zone of inhibition and prevent bond degradation caused by the formation of matrix metalloproteinase (MMP's).

INDICATIONS FOR USE:

- Desensitizing
- Rewetting tooth surface prior to bonding
- Preventing bond degradation

CONTRAINDICATIONS:

Patients with a known sensitivity, allergy, or allergic reaction to the device materials or their analogs should not use these devices.

GENERAL INSTRUCTIONS:

NOTE: ALWAYS SHAKE BOTTLE VIGOROUSLY FOR 3 SECONDS BEFORE DISPENSING.

MODE: TOTAL ETCH

1. Etch the tooth surface as instructed by the adhesive manufacturer.
2. Rinse away the extra etch and dry the tooth thoroughly for at least 3 seconds.
3. Using a small micro-applicator, apply a uniform coat of Desensitizer to the entire preparation. Allow to dwell for 5 seconds, dry briefly, leaving a shiny surface. Surface should not have any pooled liquid.
4. Continue to apply the adhesive manufacturer's materials as prescribed in its instructions.

MODE: SELECTIVE ETCH

1. Etch the enamel surface as instructed by the adhesive manufacturer.
2. Rinse away the extra etch and dry the tooth thoroughly for at least 3 seconds.
3. Using a small micro-applicator, apply a uniform coat of Desensitizer to the entire preparation. Allow to dwell for 5 seconds, dry briefly, leaving a shiny surface. Surface should not have any pooled liquid.
4. Continue to apply the adhesive manufacturer's materials as prescribed in its instructions.

DESENSITIZING CROWN PREPARATIONS:

1. Prior to cementation of the temporary, dry the tooth aggressively for at least 3 seconds.
2. **NOTE: SHAKE BOTTLE VIGOROUSLY FOR 3 SECONDS BEFORE DISPENSING.**
3. Apply 2 coats of Desensitizer, allow to dwell for 5 seconds, briefly dry.
4. For maximum effect, repeat the application several times, i.e. aggressively dry the prepared tooth, apply 2 coats of Desensitizer, allow to dwell for 5 seconds, dry again.

DESENSITIZING CROWN PREPARATIONS (continued):

5. The clinician may either choose to proceed to cement the temporary crown or apply a product of choice to seal the crown preparation and then cement the temporary.

TREATING SENSITIVE DENTIN:

1. Dry the dentin as much as possible.
2. Apply a brushful of Desensitizer.
3. Allow to dwell 5 seconds.
4. Dry.
5. Apply another brushful of Desensitizer.
6. Dry.
7. Repeat until sensitivity disappears.

NOTE: If using with a single bottle adhesive in strict self-etch mode (no etching), or with a resin modified glass ionomer, a desensitizer without HEMA should be used.

PRECAUTIONS:

1. Do not breathe dust/fume/gas/mist/vapors/spray.
2. Wash hands thoroughly after handling.
3. Wear protective gloves and eye protection.
4. Store in a well-ventilated area.
5. Keep cool, do NOT store in direct sunlight.

STORAGE:

Protect from sunlight. Store between (2 °C/36 °F) to (30 °C/86 °F).
See individual component labels for specific expiration dates.

NOTE: See PattersonDental.com for SDS.

WARNING:

Causes serious eye irritation. Wear eye/face protection. Prolonged or repeated skin contact may cause severe irritation.

FIRST AID:

IF IN EYES: Rinse cautiously with water for several minutes, also under eye lids. Remove contact lenses, if present and easy to do so.

IF INHALED: Move to fresh air and keep at a rest position comfortable for breathing.

IF INGESTED: Do not induce vomiting. Rinse mouth, drink plenty of water. Call a physician or Poison Control Center immediately.

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Patterson Dental Supply, Inc.