

retract tissue and control bleeding.



And save time.

Unlike gingival retraction cords that can be difficult and time-consuming to place, the 3M Astringent Retraction Paste is a fast, convenient and effective solution for gingival retraction. With its extra-fine tip, the 3M Astringent Retraction Paste delivers a 15% aluminum chloride astringent paste right into the sulcus, pushing the gingival tissue away from the tooth and allowing you to capture a detailed impression of the preparation margin.



Indications

3M[™] Astringent Retraction Paste is indicated for the temporary tissue retraction of the marginal gingiva to provide a dry sulcus when the periodontium is healthy, such as:

- Taking impressions (material-based or digital)
- Preparation of temporary casts
- Preparation of Class II and V fillings

The product must not be used in patients suffering from a diseased periodontium, open furcations or exposed bone.



Simple steps to a clean and dry working field



The 3M™ Astringent Retraction Paste fits most composite dispensers. Extrude a small amount of paste and discard.



Insert retraction capsule tip into the sulcus. The tissue is mechanically retracted.



Slowly and steadily, inject astringent retraction paste into sulcus. Completely fill the sulcus.



OPTIONAL: Procedure with cordsFor greater gingival deflection, the astringent retraction paste can be used in combination with retraction cords.



Leave astringent retraction paste on for a minimum of 2 minutes.



Completely remove astringent retraction paste with air-water spray and suction.

Ordering information

Item#	Product Information
56944	3M™ Astringent Retraction Paste 25 Capsules Refill Pack
56945	3M™ Astringent Retraction Paste 100 Capsules Value Pack

Note: The content of one unit-dose capsule is sufficient to treat up to three (3) teeth. The capsules are for single use and must not be used on more than one patient.



3M

3M Oral Care 2510 Conway Avenue St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249 Web 3M.com/dental 3M Canada Post Office Box 5757 London, Ontario N6A 4T1 Canada

Phone 1-888-363-3685

Customer Care Center: 1-800-634-2249 3M.com/greatimpressions

3M is a trademark of 3M. Used under license in Canada. All other trademarks are property of their respective owner. Please recycle. Printed in U.S.A. © 3M 2018. All rights reserved. 70-2013-0502-9 (Rev. 1/2018)