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N° d'urgence : 800-424-9300

Shake Before Use - EPA pH Compliant 6-8

Super Concentrated Enzyme(s) Formula eliminates (Biofilm Plaque), organic solids, scum, odors, "Daily Trap Hand Cleaning" & improves vacuum suction. Environmentally friendly & safe for all plastic parts and pipes.

Daily Maintenance Protocol - At closing (per 2 operatory) add (1 dispenser pump) 1oz. (30 mL) Patterson Evacuation Cleaner to 64 oz. (2 L) of water, stir well. Aspirate 16 oz. (.5 L) of solution into each valve, one valve at a time, 32 oz. (1 L) per operatory at closing.

IMPORTANT: Check traps weekly, daily trap cleaning will no longer be needed. Reseal lid to ensure freshness & store out of direct sunlight.

Rinse Dispenser After Use, Pour Residual Mixture Down Sink Drain.

WARNING

Contains: Citric acid; Sodium benzoate; Sodium propionate

Hazard Statements: Causes skin and eye irritation.

Precautionary Statements: Read label before use. Wear protective gloves/protective clothing/eye protection/face protection. Wash skin thoroughly after handling.

IF ON SKIN: Take off contaminated clothing and wash it before reuse. Wash skin with plenty of water and soap. If skin irritation occurs seek medical advice or attention.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists seek medical advice or attention. Specific treatment (see supplemental first aid instructions on this label). Dispose of contents or container in accordance with federal, provincial and local regulations.

Patterson Dental Supply, Inc. 1031 Mendota Heights Road, Saint Paul, MN, 55120, USA | 800-328-5536

Emergency Phone: 800-424-9300

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Patterson Dental Supply, Inc. 1031 Mendota Heights Road, Saint Paul, MN, 55120, É.-U.