

Sirona LED The first LED to show its true colors.



- [HIGHLIGHTS](#)
- [PROPERTIES](#)
- [TECHNICAL DATA](#)
- [DOWNLOADS](#)

Sirona LED compatibility with Sirona dental units

The voltage setting in the units is preset to 3.6V by default. Normally all there is to do is to set the brightness to 100 %.

Dental unit	Instrument						SIROSONL SIROSONIC L	Brightness setting on the dental unit
	Motors		EL 1	BL	BL ISO	Highspeed handpiece		
	SL	SL ISO						
M1-90, M1-94	■	■				■		100 %
M1-96	■	■				■	■	100 %
Siemens Sirona E	■	■				■	■	Standard (90 %)
C2, C3, C4	■	■	■			■	■	100 %
C2+, C3+, C4+, C5+	■	■	■			■	■	100 %
C6, C8, C8+, ProFeel+	■	■	■			■	■	Standard (90%)
TENEO	■	■		■	■	■		100 %

Dental unit	Instrument						SIROSONL SIROSONIC L	Brightness setting on the dental unit
	Motors							
	SL	SL ISO	EL 1	BL	BL ISO	Highspeed handpiece		
SINIUS				■		■		100 %

Sirona products with Sirona LED

Name	REF
Sirona LED	63 14 558
Quick coupling R	63 19 706
Motor hose SL D+ C2/C4	63 19 607
Motor hose SL D+ 1660mm	63 19 615
Motor hose SL D+ C3/C6	63 19 623
Motor hose SL D+ C8	63 19 631
Motor hose SL D+ C8+/Profeel	63 19 649
Sirosonic L hose C2/C4	63 19 664
Sirosonic L hose C3+/C6	63 19 672
Sirosonic L hose C8+/Profeel	63 19 680
Sirosonic L hose 1660mm	63 19 698
Motor BL	63 19 714
Motor BL ISO	63 19 722