

gloveon Eureka



Nitrile Exam Gloves Powder Free, Standard Cuff

To don **Eureka** is to experience absolute dexterity, comfort fitting and all the properties of a quality nitrile glove, everytime.



GloveOn Eureka		
Length (mm)	≥ 230	
Thickness Measurements (mm)		
Palm (centre of Palm)	≥ 0.05	
Finger (13mm ± 3mm from tip)	≥ 0.07	
Physical Properties	Before Ageing	After Ageing
Tensile Strength (MPa)	≥ 14	≥ 14
Elongation (%)	≥ 500	≥ 400
Inspection Levels & AQL	Inspection Level	AQL
Watertightness	G1	1.50
Physical Dimensions	S2	4.00
Tensile Strength	S2	4.00
Visual Inspection (Major)	S4	2.50
Visual Inspection (Minor)	S4	4.00
Particulate Residue	N = 5	≤ 2mg/glove

Chemotherapy Drugs and Concentration (Tested for Resistance to Permeation by Chemotherapy Drugs as per ASTM D6978 - 05-Test Report PN 116669)	Minimum Breakthrough Detection Time (minutes)
Carmustine (BCNU), 3.3mg/ml (3,300 ppm)	1.7 minutes
Cisplatin, 1.0mg/ml (1,000 ppm)	>240 minutes
Cyclophosphamide (Cytosan), 20.0mg/ml (20,000 ppm)	>240 minutes
Dacarbazine (DTIC), 10.0mg/ml (10,000 ppm)	>240 minutes
Doxorubicin Hydrochloride, 2.0mg/ml (2,000 ppm)	>240 minutes
Etoposide (Toposar), 20.00mg/ml (20,000 ppm)	>240 minutes
Fluorouracil, 50.0mg/ml (50,000 ppm)	>240 minutes
Methotrexate, 25.0mg/ml (25,000 ppm)	>240 minutes
Mitomycin C, 0.5mg/ml (500 ppm)	>240 minutes
Paclitaxel (Taxol), 6.0mg/ml (6,000 ppm)	>240 minutes
Thiotepa, 10.0mg/ml (10,000 ppm)	10.8 minutes
Vincristine Sulfate, 1.0mg/ml (1,000 ppm)	>240 minutes

WARNING: Carmustine and Thiotepa, at the tested concentration, degraded Eureka nitrile glove at 1.7 minutes and 10.8 minutes, respectively. The safe use of gloves in chemotherapy treatment is solely the decision of clinicians authorised to make such decision.

REORDER CODE

NTR61XS	X-SMALL
NTR61SS	SMALL
NTR61MM	MEDIUM
NTR61LL	LARGE
NTR61XL	X-LARGE

FEATURES

- Fingertip textured
- Powder free
- Not made with natural rubber latex
- Chemo drugs tested
- Lab chemical tested
- Ambidextrous
- Standard cuff
- Dark blue colour

PACKAGING

300 gloves per box for XS to L
270 gloves per box for XL
10 boxes per carton

REGULATORY COMPLIANCE

TGA - ARTG 164563, FDA 510(k),
MDD 93/42/EEC, REACH,
ROHS Directive 2002/95/EC, EC 10/2011,
REGULATION (EC) No 1935/2004,
PPE 89/686/EEC

STANDARDS

ASTM D6319, EN 455 part 1, 2, 3 & 4
(except Force At Break), EN 1186, EN 13130,
CEN/TS 14234, EN 420, EN 374 part 1, 2 & 3

MANUFACTURING ACCREDITATIONS

ISO 9001:2008
ISO 13485:2003
EN ISO 13485:2003

A brand by

mun
www.munglobal.com

Mun Australia
Toll free: 1800 456 837 (GLOVES)
info.au@munglobal.com

**Birthing Kit
Foundation**
Australia
Proud Supporter of
Birthing Kit Foundation (Australia)

gn Protection Always On