

4 reasons to use SpotIt® C&B contact finder

#1 Exact indication for adjustment of C&B work

Spot it simplifies adjustment of C&B contact points. Exact indication of contact points.



#3 Easy handling

Easy to use - no assistance needed. Time-saving and accurate.

#2 Optimal access

Optimal access thanks to straight and angled design.



#4 No mess

Clear indication, leaves no marks outside the contact area. Can be used without assistance.

SpotIt®

Prod No	Product Description	Quantity
690112	SpotIt Assortment	9+9 pcs
690110	SpotIt Refill Straight	18 pcs
690111	SpotIt Refill Angled	18 pcs

C & B contact finder for exact fitting of prosthetics.

SpotIt
Design by Dentists™



1. Place crown or bridge.



2. Push SpotIt through the contact to get the indication mark.



3. The exact indication of where trimming is needed.



4. Indication after removed bridge.

Spot it C&B contact finder

Identifying contact points on crowns and bridges has never been easy.

Before cementation of conventional crowns, bridges and implant prosthetics, it is necessary to assess the relationship of the crown to the neighboring teeth. Occlusion paper is often used to check the contact point. This method however shows a too broad and inaccurate contact point with the subsequent risk of excessive adjustment, often resulting in too large contact point.

Directa's new contact finder Spot it is a coloured marker available on straight and angled handle. As the marker traverses the contact between the prosthetic element and the adjacent tooth it leaves a clear and accurate indication of the adjustment needed for a perfect contact point. The whole operation can be performed without assistance - holding the crown with one hand and the marker with the other hand.

Identifying contact points on crowns and bridges has never been easier!

Straight



Angled

