

**Patterson® Waterline Maintenance Tablets**  
Comprimés d'entretien des conduites d'eau  
Dental Unit  
Waterline Treatment  
Entretien des conduites d'eau des unités dentaires

Inhibits the Growth of Microbial Contamination in Dental Unit Waterlines  
Empêche la croissance de la contamination microbienne dans les conduites de l'unité dentaire

**CAUTION**  
Keep Out of Reach of Children  
See back panel for additional precautionary statements.  
Net contents: 50 Tablets • 0.26 oz. total

**MISE EN GARDE**  
Garder hors de portée des enfants  
Consulter les mises en garde additionnelles au verso  
Contenu : 50 comprimés (0,26 oz au total)

Reorder Number  
N° de produit  
**041-9184**

Manufactured for:  
Patterson Dental Supply, Inc.  
1031 Mendota Heights Road  
Saint Paul, MN 55120  
PATTERSON DENTAL  
Fabriqué pour :  
Patterson Dental Supply, Inc.  
1031 Mendota Heights Road  
Saint Paul, MN 55120  
PATTERSON DENTAL  
MADE IN U.S.A. - Fabriqué aux É.-U.

**Patterson® Waterline Maintenance Tablets**

Clinical monitoring of water quality can ensure that procedures are being performed correctly and are effective. Test water quality monthly to establish effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat Initial Treatment.

PRECAUTIONARY STATEMENTS: Hazards to Humans and Domestic Animals.

CAUTION: Tablet may cause moderate eye irritation. Avoid contacting tablet with eyes or clothing. Wash thoroughly with soap and water after handling tablets and before eating, drinking, chewing gum, or using tobacco.

FIRST AID: If in Eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first five minutes and continue rinsing. Call a poison control center or doctor for treatment advice.

HOTLINE NUMBER: In case of emergency call toll free 1-866-290-1830. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

Patterson® Waterline Maintenance Tablets are designed to be continuously present in your dental unit water system.

**Patterson® Waterline Maintenance Tablets**

STORAGE AND DISPOSAL  
STORAGE: Do not contaminate water, food, or feed by storage or disposal. Store in a cool, dry place.

FOIL POUCH  
DISPOSAL: If empty, place in trash or offer for recycling if available. If partly filled, call your solid waste agency (or 1-800-CLEANUP) for disposal instructions.

OUTER CONTAINER  
Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of the empty container in trash.

.7L Tablets - Reorder # [041-9184]

Manufactured for:  
Fabriqué pour:  
Patterson Dental Supply, Inc.  
1031 Mendota Heights Road  
Saint Paul, MN 55120  
PATTERSON DENTAL

EPA Reg. No. 84020-1-43100  
EPA Establishment No. 84065-MO-001

MADE IN U.S.A. - Fabriqué aux É.-U.  
MSN XXX-XXX

**Patterson® Waterline Maintenance Tablets**

**ACTIVE INGREDIENT:**  
Silver 0.477%

**OTHER INGREDIENTS: 99.523%**  
TOTAL 100.000%

**CAUTION**  
Keep Out of Reach of Children  
See back panel for additional precautionary statements.

Net contents: 50 Tablets • 0.26 oz. total  
MADE IN U.S.A. - Fabriqué aux É.-U.

Reorder Number  
N° de produit  
**041-9184**

**.7L Tablets**

**Patterson® Waterline Maintenance Tablets**

DIRECTIONS FOR USE: IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING. READ ENTIRE LABEL BEFORE USING THIS PRODUCT.

INITIAL TREATMENT: Before starting a treatment program with Patterson® Waterline Maintenance Tablets, flush lines with a waterline cleaner or another product recommended by the equipment manufacturer to remove line buildup or deposits. When flushing dental unit water lines, always follow a recognized line flushing/cleaning protocol. Repeat flushing to clean lines every 2 months or more frequently when water quality results indicate.

ROUTINE TREATMENT:

- Place tablet or tablets (see table) in empty dental unit water bottle every time you refill the water. Patterson® Waterline Maintenance Tablets are intended for use with potable water (including tap or distilled).
- Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.
- Patterson® Waterline Maintenance Tablets may be left in the waterlines overnight and for extended periods of non-use. Use water solution until it is expended or up to 28 days. It is not necessary to purge lines at the end of each day.

Water Volume	Tablets
700-750ml (24-25 fl. oz.)	1
1500ml (51 fl. oz.)	2
2000ml (68 fl. oz.)	3