



HIGH-LEVEL DISINFECTANT

SANI Glut™

Glutaraldehyde 3%



When safety counts and minutes matter.

SANI Glut™ Glutaraldehyde (3%) with activator is a reusable sterilizing and high-level disinfection solution. **SANI Glut** is non-corrosive and is compatible with immersible, heat sensitive instruments and devices. Once activated, **SANI Glut** can be used up to 28 days* **SANI Glut** provides high-level disinfection in 25 minutes and sterilization in 10 hours



SANI Glut™
Glutaraldehyde 3%

28 Day Reuse
25 Minute High Level Disinfection
10 Hour Sterilization

Reusable Sterilizing and High Level Disinfecting Solution

| | |
|-------------------|--------|
| Glutaraldehyde | 3.0% |
| Other Ingredients | 97.0% |
| Total | 100.0% |

Keep Out Of Reach Of Children
DANGER: See side panel for additional precautionary statements

Contents: 1 Gallon (128 FL. OZ./3.8 L)

CROSSTEX SPS
A CANTEL MEDICAL COMPANY medical

The *SANI Glut*TM Advantage

FAST ACTING AND EFFECTIVE

*SANI Glut*TM Glutaraldehyde is fast acting and useful against a variety of microorganisms. It is readily biodegradable, formaldehyde-free and performs more effectively than products that use formaldehyde, glyoxal or other corrosive materials.

FEATURES & BENEFITS

- **EFFECTIVE** – achieves high-level disinfection in 25 minutes and sterilization in 10 hours.
- **CONVENIENT** – once activated, ready to use.
- **ECONOMICAL** – reusable for up to 28 days*.
- **EXCELLENT MATERIALS COMPATIBILITY** – non-corrosive and compatible for use on heat sensitive instruments and devices.
- **EASY ACTIVATION** – activate *SANI Glut* by adding the entire contents of the activator vial which is attached to the *SANI Glut* container.

TECHNICAL INFORMATION

- In-Use Concentration: 3% Glutaraldehyde
- High-Level Disinfection: 25 minutes at 25°C
- Sterilization: 10 hours at 25°C
- Use Life: Up to 28 days* once activated
- Use Conditions: Designed for manual use
- Shelf Life: 2 years
- Storage: 15-25°C
- Disposal: Drain as per facility policy. Flush thoroughly with water



* Solution needs to be tested with the appropriate test strip prior to each use to verify effective disinfection concentration.

BY THE NUMBERS

28 DAY reuse period*

25 MINUTE minimum high-level disinfection time at 25°C

10 HOUR sterilization at 25°C



Ordering Information

| Description | Quantity | Ref.# |
|---|--------------------------|-------|
| <i>SANI Glut</i> TM Glutaraldehyde 3% with activator | 1-gallon bottle | JGLUT |
| SANI HLD Holder | Holds 1 gallon of liquid | JHOLD |

CROSSTEX | PROTECTS[®]
A CANTEL MEDICAL COMPANY

Manufactured for Crosstex
10 Ranick Road, Hauppauge, NY 11788-4209 USA
Tel: 631-582-6777 | 888-276-7783 | Fax: 631-582-1726
E-mail: crosstex@crosstex.com | crosstex.com

California • Colorado • Georgia • New York • Pennsylvania • Argentina • Japan • Singapore • The Netherlands