

Maxcem Elite Chroma – Technique Card

Picture examples make cleanup easy!

Optimal Time for Cement Removal



Placement

- Seat the restoration on the preparation. Hold it in place with moderate pressure.
- Allow the cement to flow from all the margins. Cement color will change from pink to white¹ as it cures.



Cleanup

- Once pink color is diminished, remove excess cement while maintaining pressure on the restoration. Excess cement is best removed in its gel state (gel state is achieved in 2-3 minutes if self-cure, or 2-3 seconds tack-curing).



¹ For yellow cement shade, color will change from pink to ivory.

