Instructions for Use / Instrucciones de uso / Instructions d'utilisation

Step 1:
Select an impression tray.

Step 2:
Try the tray in the mouth and assess for the tray size. There should be 1/4 in. of space allowance for the impression material.

Step 3:
Assess whether the tray requires reshaping due to arch shape or excess peripheral borders.

Step 4:
If adjustment is required dip the selected part of the tray to be adjusted in a hot water bath at 158°F or 70°C for 20 seconds, the material will become soft and pliable at this stage.

Step 5:
Mould the tray to the desired shape, retain in this position until hard (approx. 10 seconds) allow the material to cool or dip in cold running water.

Step 6:
The material will retain the moulded shape and maintain its strength. This process may be repeated until the desired shape is achieved.

Step 7:
The modified tray can now be used to take the impression.

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Step 4 / Paso 4 / Étape 4

Step 5 / Paso 5 / Étape 5

Tray Adhesive Application Instructions

Instrucciones de aplicación del adhesivo para cubetas / Instructions pour l'application de l'adhésif pour porte-empainte

Step 1
Be sure that all surfaces of the impression tray are oil-free, clean and dry. NOTE: If the disinfection of the impression tray cannot be ensured, the tray adhesive must be poured into a disposable container and applied using a disposable brush.

Step 2
Apply a thin layer of AccuDent XD Tray Adhesive using a brush to all surfaces of the impression Tray (metal or plastic) that will come into contact with the impression material.

Step 3
Allow the tray adhesive to dry for approximately 3-5 minutes (or blow dry with compressed oil-free air).

Step 4
Replace the bottle cap immediately.

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