United States
175 Pineview Drive
Amherst, New York 14228
800-533-6825
Fax: 716-691-2285
www.ivoclarvivadent.com

Canada
#1 - 6600 Dixie Road
Mississauga, Ontario L5T 2Y2
800-263-8182
Fax: 905-670-3102
www.ivoclarvivadent.com

Ivoclar Vivadent AG,
FL-9494 Schaan/Liechtenstein

100% CUSTOMER SATISFACTION
GUARANTEED!

ivoclarvivadent.com

©2014 Ivoclar Vivadent, Inc.
AccuDent and Ivoclar Vivadent are
registered trademarks of Ivoclar Vivadent.
Rev.1 7/2014

AccuDent® XD

Alginate Tray Adhesive

Instructions for Use
Mode d'emploi
Instrucciones de Uso

Adhesive for alginate impression materials
Adhesivo para materiales de impresión de alginate
Colle pour matériaux d'empreinte alginate

Rx
For dental use only!

Made in USA for
Ivoclar Vivadent, Inc.

Clean up with Isopropyl Alcohol.

Warnings
— Isopropyl alcohol and toluene are hazardous to health
— Avoid contact between AccuDent XD Tray Adhesive and skin, mucous membrane, and eyes; if contact should occur, rinse thoroughly with running water for 10 minutes; with the affected eye open. Consult ophthalmologist. If the material comes into contact with the skin, immediately wash affected areas with water.
— Do not inhale vapor.
— Close container after use.
— Keep the product away from flame and sparks.
— Attention: May stain clothing.

Storage
— Do not use AccuDent XD Adhesive after the indicated date of expiration.
— Close bottle immediately after use.
— Do not use material if the individual components show clear signs of change of consistency;
— Storage temperature: 0 – 30° C (32 – 86° F)
— Shelf life: See expiration date on label and packaging.

Keep out of the reach of children!

Caution
Contents are flammable. Keep out of reach of children and from sparks or open flame. Do not ingest. See MSDS for chemical make-up at www.ivoclarvivadent.com.

For use in dentistry only!
The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and data constitute no warranty of attributes and are not binding.

Description
Self-curing, single-component adhesive for alginate based impression materials.

Composition
Contains polyamide resin and solvent.

Indication
AccuDent XD Tray Adhesive is specifically intended for the adhesion of cured alginate impression materials to standard metal and plastic trays.


Contraindication
— If the patient is known to be allergic to any of the ingredients in AccuDent XD Tray Adhesive.
— Adhesive should only be used with disposable trays.

Side Effects
AccuDent XD Tray Adhesive may cause a sensitization reaction in patients with a longer be applied.

Interactions
None

Application

1. Be sure that all surfaces of the impression tray are oil-free, clean and dry. NOTE: If the disinfection of the impression tray cannot be ensured, the tray adhesive must be poured into a disposable container and applied using a disposable brush.
2. Apply a thin layer of AccuDent XD Tray Adhesive using a brush to all surfaces of the impression tray (metal or plastic) that will come into contact with the impression material.
3. Allow the tray adhesive to dry for approximately 3-5 minutes (or blow dry with compressed oil-free air).
4. Replace the bottle cap immediately.

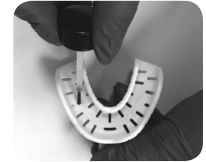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