



Steam Tape Instructions For Use:

1. Open the plastic wrapper and place in a compatible tape dispenser.
2. Unwind a section of the tape approximately 20-30 cm (8-12 inches) long.
3. Place one end of the indicator on the package and apply finger pressure across the length of the tape to secure the pack.
4. Do not stretch the tape by pulling on it, which may cause bunching of the material.
5. Repeat steps 2 and 3, utilizing two strips of tape to secure the package/tray.
6. After sterilization allow pack / wrapped tray to cool 30 minutes prior to breaking the seal. The chemical indicator lines shall turn dark brown/black indicating a sterilization process occurred.
7. Do not remove chemical indicator tape in operating room because linting may occur.
8. Remove sterilization tape from linens and untreated woven fabrics, to prevent adhesive from gumming or sticking.

Storage Conditions: Store at controlled room temperature as defined by the United States Pharmacopeia*. Controlled room temperature is thermostatically controlled between 20 and 25°C (68-77°F), with a mean kinetic temperature of not more than 25°C, with excursions to 15°C-30°C (59-86°F) allowed. Reference the USP for the complete definition.