

## **Flip Style High Volume Suction Valves**

### **Note:**

- Follow manufacturers' instruction and guidelines for maintenance of aspirator valves.

### **Autoclave Parameters:**

- Do not exceed 275° F or 135° C.
- Sterilizing aspirator valves in an ultrasonic bath is NOT recommended.

### **Daily Maintenance:**

- To minimize biofilm buildup, flush suction valve thoroughly with water between each patient.
- Disassemble suction valve and clean thoroughly with water and/or disinfecting solution.

### **CAUTION:**

- The use of Sodium Hypochlorite or any solutions containing bleach will result in permanent damage of your aspirator valve.