



NoCord™ VPS

One Step, Self-Retracting, Impressioning System



A unique impressioning system that makes impression taking easier, more predictable and more assured.

Introducing **Centrix NoCord™ VPS!** A revolutionary one step, self-retracting, impressioning system designed specifically for the wash and tray technique. NoCord™ Wash and NoCord™ MegaBody™ Tray Materials are uniquely designed to produce highly accurate dental impressions without the need for retraction cords and pastes.

Revolutionary Material.

- **NoCord Wash** is the first wash material to incorporate a hemostatic agent to control bleeding and fluid flows.
 - 15% alum (ammonium aluminum sulfate) effectively closes cut capillaries, controlling bleeding
- The unique **NoCord MegaBody Tray Material** is ‘stiffer’ to drive the wash into the sulcus to displace gingiva and completely capture prep margins with NoCord Wash Material.
 - No retraction cord means no gingival trauma
 - More comfortable for patients, easier and less time consuming for you

Saves Real Time, Real Money.

- All-in-one system that retracts, controls bleeding and takes a clear, final impression.
- Eliminates the time and costs to use retraction cord or paste and to apply an astringent.


Technique Friendly.

- There’s no need to learn a new procedure.
- Simply use your traditional, one step wash and tray technique.

Brilliant Results¹.

- Clear, accurate, easily readable margins in your impressions, every time.
- Restorations that fit, every time. No adjustment needed.

Perfect Impression

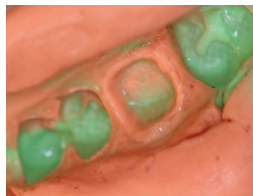


Clean Open Margin



Photos by Dr. Jon A. Heezen, DDS

Perfect Impression

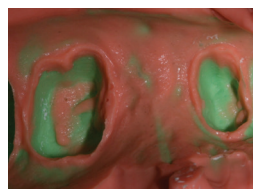

Clean Open Margin




Photos by Dr. Frank J. Milnar, DDS

¹Data on File


One Step, Wash & Tray Technique:

1




Prepare tooth as usual. Dry prep with dressing or air syringe.

2




Syringe copious amount of NoCord™ Wash Material into the gingival sulcus and around the prep.

3




Fill impression tray with MegaBody™ Tray Material. A walled, dual or full arch tray like the Centrix 3.n.One™ is recommended.

4




Seat MegaBody Tray Material over the unset NoCord Wash Material. The unique stiffness of MegaBody drives the wash material into the sulcus for complete margin capture.


5



Have patient bite down until material is set. Remove from patient's mouth at 3:45 minutes. Full set is 4:45 minutes.




NoCord VPS gently displaces gingiva from around the tooth, creating sufficient space for NoCord Wash to completely capture margin detail.

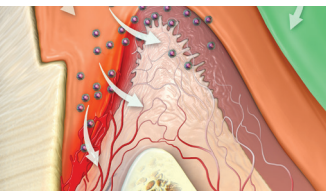

NoCord Wash contains 15% alum to constrict capillary oozing, control bleeding, and displace gingival crevicular fluids.

6



Check impression. Fabricate final restoration. Easy, accurate impressions!

Photos courtesy of Dr. Jon A. Heezen DDS, Tracy, MN

Product/Material	Color	Technique	Mixing Time	Working Time	Intraoral Time	Total Set Time
NoCord™ Wash	Coral Orange	Wash & Tray	Auto	1:00	3:45	4:45
NoCord™ MegaBody™ Tray	Seafoam Green	Wash & Tray	Auto	1:00	3:45	4:45

Ordering Information:

Introductory Kit

REF 310280 NoCord™ VPS Introductory Kit

- 1 NoCord Wash Material (50mL cartridge)
- 2 NoCord MegaBody Tray Material (50mL cartridges)
- 12 SuperMixer mixing nozzles, teal hub
- 12 SuperMixer mixing nozzles, yellow hub with 18ga green tip

2-Pack Refills

REF 310282 NoCord™ Wash Material

- 2 NoCord Wash Material (50mL cartridges)
- 6 SuperMixer mixing nozzles, yellow hub with 18ga green tip

REF 310281 NoCord™ MegaBody™ Tray Material

- 2 NoCord MegaBody Tray Material (50mL cartridges)
- 6 SuperMixer mixing nozzles, teal hub

10-Pack Refills

REF 310284 NoCord™ Wash Material

- 10 NoCord Wash Material (50mL cartridges)
- 12 SuperMixer mixing nozzles, yellow hub with 18ga green tip

REF 310283 NoCord™ MegaBody™ Tray Material

- 10 NoCord MegaBody Tray Material (50mL cartridges)
- 12 SuperMixer mixing nozzles, teal hub

System Refill Kit

REF 310285 NoCord™ VPS System Refill Kit

- 6 NoCord Wash Material (50mL cartridges)
- 12 NoCord MegaBody Tray Material (50mL cartridges)
- 12 SuperMixer mixing nozzles, teal hub
- 12 SuperMixer mixing nozzles, yellow hub with 18ga green tip

Suggested Accessories:

- REF 360006** AccessBLUE™ VPS Quick Bite Registration Material (2x50mL cartridges)
- REF 910628** SuperMixer mixing nozzles, yellow hub combo, 50 (use with wash materials)

- REF 360010** SuperMixer mixing nozzles, teal hub, 24 (use with tray materials)
- REF 360013** 3.n.One™ Quadrant Trays, 35
- REF 360011** 3.n.One Anterior Trays, 35

Ordering is easy. Call 800.235.5862 or contact your preferred dealer.

Visit CentrixNoCordVPS.com for more information.

