Use only as directed. For use only as directed. For use only as directed.

G-Multi PRIMER

PRIMER FOR GLASS CERAMICS, HYBRID CERAMICS, ZIRCONIA, ALUMINA, COMPOSITES AND METAL BONDING

G-Multi PRIMER is a priming agent designed to treat the adherent surface of glass ceramics, hybrid ceramics (e.g. CERASMART), zirconia, alumina, composite and metal restorations. This is to be used before luting procedures. The material as well as resin-based cements in case of luting procedures or direct composites in case of repairs.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Priming of adherent surface of indirect restorations made of glass ceramics, hybrid ceramics (e.g. CERASMART), zirconia, alumina, composite and metal before luting procedures.
- Intra-oral and extra-oral repair of fractured indirect restorations made of ceramics, hybrid ceramics, zirconia, alumina, and composites.

CONTRAINDICATIONS
Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

DIRECTIONS FOR USE

- Treat adherent surface of restoration according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. We recommend cleaning in an ultrasonic bath or with alcohol, then rinse and dry with an oil-free air syringe.
- Dispense one drop of G-Multi PRIMER into a syringe.
- Apply a thin layer to the adherent surface of the restoration using a micro-tip applicator and dry with an air syringe.
- Apply a thin layer of luting cement to the primed surface of the restoration. For these steps, refer to the manufacturer's instructions and corresponding luting cement.
- For Intra-oral Repair
 - Roughen adherent surface using a coarse diamond bur / a carbondurum point or with a diamond bur.
 - Treat adherent surface of the restoration according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.
 - Dispense one drop of G-Multi PRIMER into a syringe.
 - Apply a thin layer to the fractured surface of the restoration using a micro-tip applicator and dry with an oil-free air syringe.
 - Apply a thin layer of luting cement to the primed surface of the restoration. For these steps, refer to the manufacturer's instructions and corresponding luting cement.
- For Intra-oral Repair
 - Roughen adherent surface using a coarse diamond bur / a carbondurum point or with a diamond bur.
 - Treat adherent surface of the restoration according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.
 - Dispense one drop of G-Multi PRIMER into a syringe.
 - Apply a thin layer to the fractured surface of the restoration using a micro-tip applicator and dry with an oil-free air syringe.
 - Apply a thin layer of luting cement to the primed surface of the restoration. For these steps, refer to the manufacturer's instructions and corresponding luting cement.

STORAGE
Recommended for optimal performance, store at room temperature (4 - 25°C / 39,2 - 77,0°F).

Use only as directed. For use only as directed.

See expiration date.

CAUTION

- Avoid protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- G-Multi PRIMER is flammable and volatile. Do not allow open flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
- Avoid contact with oral tissues, immediately remove with a cotton pellet or a sponge. Rinse immediately after luting or repairing.
- In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Do not mix with any other primers.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced discontinue the use of the product and refer to a physician.
- Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas <http://www.gceamerica.com>
- This can also be obtained from your supplier.

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