

*CAUTION: Some special care needs to be exercised in cleaning and sterilizing this coated aluminum instrument. Clean by hand. Do not clean in an ultrasonic cleaner. Sterilize in autoclave for at least 4 min. at 270°F/132°C or 30 min. at 250°F/121°C. Do not heat above 350°F/177°C. Recommend 30-minute dry time after sterilization cycle. The use of iodophor or phenol solutions is not recommended.*

*MISE EN GARDE: Une attention particulière doit être portée au nettoyage et à la stérilisation de cet instrument d'aluminium traité en surface. Nettoyer à la main. Ne pas nettoyer dans une unité de nettoyage ultrasonique. Stériliser en autoclave durant au moins 4 minutes à une température de 132°C/270°F ou durant 30 minutes à une température de 121°C/250°F. Ne pas chauffer à plus de 177°C/350°F. Temps de séchage recommandé de 30 minutes suivant le cycle de stérilisation. L'utilisation d'iodophores ou de solutions de phénol n'est pas recommandée.*