


Zoral Scaler Soution®

Mechanism of Action

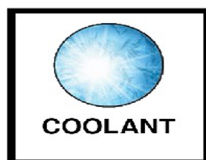

FDA registered.

Designed exclusively for all powered dental scalers.

- **Zoral Cools**; aiding in ease of dental scaler application.
- **Zoral Lubricates** the dental scaler and soft tissue for optimal treatment.
- **Zoral is CHG Fortified**; promoting soft tissue healing & health.

How to use Zoral?

1. Pour Zoral into the scaler's empty reservoir. Do not dilute Zoral.
 2. Activate the Scaler. Adjust the power and flow to the desired level.
 3. Begin dental scaling procedures.
- EFFECTIVE & EASY TO USE.**