


# Processor Maintenance As Easy As...

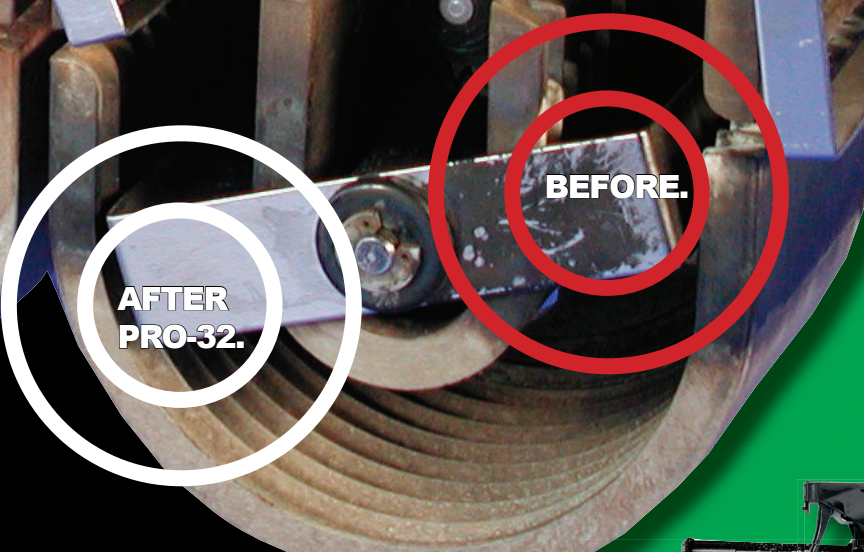

**FREE**  
Harley Davidson  
Safety Glasses

## STEP 1 Daily Cleaning

### CLEAN IMAGE

**Clean Crisp Film Every time. Eliminates Grainy Film.**

- Process a new Clean Image Sheet as the first film of the day to remove artifacts that stick to rollers overnight.
- After each cleaning process a new Clean Image Sheet to remove any loosened debris.
- Replenish with Panopro Developer and Fixer.



## STEP 2 Weekly Cleaning

### CLEAR IMAGE

**Insures Quality X-Rays.**

- Remove rollers and rinse well with warm water.
- Apply Clear Image to each roller or peri-type transport with the foaming applicator.
- Wait 2-3 minutes, scrub rollers with a green 3M Scotch-Brite pad and rinse well with warm water. For peri-type: use a stiff brush to clean the transport grooves.
- Return rollers or peri-type transport and replenish with Panopro Developer and Fixer.
- Process a new Clean Image Sheet as the first film.
- Apply Clear Image monthly to fixer rollers, wash rack, and tanks.

## STEP 3 Monthly Cleaning

### PRO-32

**20-Min. Soak Evaporates Silver. No Odor, No Fumes, No Hazardous Shipping Fees.**

- Rinse rollers and tanks with warm water.
- Prepare Pro-32 solution. Mix 3.5oz. per gallon of water in rubber tub or well-rinsed developer tank. Safety First: wear safety glasses, nitrile gloves, and long sleeves.
- Soak rollers or peri-type transports in Pro-32 solution for 20-45 minutes.
- Rinse and soak rollers or peri-type transports in fresh water for 10 minutes.
- Process a new Clean Image Sheet as the first film.

**Once Clean, Keep Your Processor Running Cleaner Longer with Panopro Developer & Fixer.**



Dark chemistry is a visible sign of silver buildup, oxidation, and poor quality x-rays.

\* **PANOPRO** stays fresh longer. Lighter chemistry is proof of a cleaner processor and sharper x-rays.