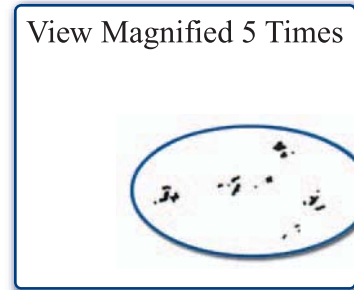

CLEAN IMAGE

Transport Roller Clean-Up Sheets

- * *Replaces your clear clean-up sheet.*
- * *Immediate clean, crisp film every time.*
- * *Bright white sheet visibly shows the removal of artifacts.*
- * *Moisture-activated cleaning gel removes artifacts that accumulate overnight.*
- * *Reduces processor downtime needed to clean rollers.*
- * *Eliminates film waste and retakes due to buildup.*
- * *Use daily and after each cleaning.*



See the Dirt!

Directions: Set processor to the regular developing cycle. Send the **CLEAN IMAGE** sheet through the processor with the shiny side up to clean the upper rollers, then shiny side down to clean the lower rollers. Discard the used **CLEAN IMAGE** sheet.

Southland Distribution & Sales
phone: 800-880-0240 or 706-868-0105, fax: 706-650-7468
www.sdssouthland.com, e-mail: info@sdssouthland.com

SDS
SOUTHLAND DISTRIBUTION & SALES
Mfg. # CI0810

Processor Maintenance

*Your
Total
Solution!*



1 DAILY - Clean Image Sheets

1. Process a new **CLEAN IMAGE** Clean-Up Sheet as the first film of the day to remove artifacts that build up overnight.
2. Replenish with **PANOPRO** developer and fixer.



2 WEEKLY - Clear Image Foam

1. Remove rollers and rinse thoroughly with warm water.
2. Apply **CLEAR IMAGE** to all rollers with the foaming applicator.
3. Wait 2-3 minutes, scrub rollers and rinse thoroughly with warm water.
4. Return rollers and replenish with **PANOPRO** developer and fixer.
5. Process a new **CLEAN IMAGE** Clean-Up Sheet.



CLEAR IMAGE
Weekly



CLEAR IMAGE
Weekly



PRO-32
Monthly



PRO-32
Monthly

3 MONTHLY - Pro-32

1. Safety First: Wear safety glasses, nitrile gloves, and long sleeves.
2. Prepare **PRO-32** solution (3.5 oz. per gallon of water).
3. Soak 20-45 minutes.
4. Rinse and soak rollers in fresh water for 10 minutes.
5. Apply **CLEAR IMAGE** to fixer rollers and wash rack.
6. Wait 2-3 minutes, scrub rollers and rinse thoroughly with warm water.
7. Apply **CLEAR IMAGE** to tanks.
8. Wait 2-3 minutes, scrub tanks and rinse thoroughly with warm water.
9. Return rollers to processor. Replenish **PANOPRO** developer and fixer.
10. Process a new **CLEAN IMAGE** Clean-Up Sheet.

Peri Type: Soak 20-45 minutes. Rinse and soak in fresh water for 10 minutes.