


Bands

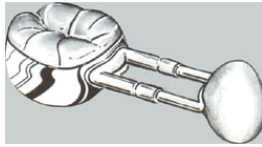

The method of choice for saving space in young patients who experience early loss of primary molar. Now available on standard molar preformed bands. In-office, one treatment.


1. **Select the proper size band and trial fit to the tooth, then remove.**



2. **Slide wire into pre-welded tube, determine the proper length, cut wire, deburr and lightly crimp.**



3. **Trial fit assembly. Crimp tubes over wire. Seat the Space Maintainer Band and cement in place.**

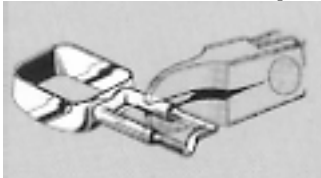

4. **Appliance should be free from teeth in contact from opposing arch.**



Space Maintainer Bands are provided only with a pre-welded tube ensuring consistency, comfort, speed in handling, and accuracy in usage.