GOCCLES®

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 **GOCCLES®** is a new innovative device that permits an easy, non invasive and non-painful examination of the oral cavity, able to reveal cancerous and pre-cancerous lesions.

INTENDED USE

GOCCLES is intended to be used by qualified health-care providers to enhance the identification and visualization of the oral mucosal abnormalities that may not be apparent or visible

MECHANISM OF ACTION

This device allows the examination of auto-fluorescence of the oral cavity in order to enhance the identification and visualization of the oral mucosal abnormalities that may not be apparent or visible to the naked eye, such as oral cancer and premalignant dysplasia. The G.o.c.c.i.e.s.-based examination is thus recommended as complementary methodology to be used together with the standard naked eye-based examination.

While the visualization of the oral cavity with the naked eye can make out only a small fraction of the spectral characteristics that differentiate healthy tissue from the diseased one, the optical methods based on tissue auto-fluorescence improves the ability to detect cancerous lesions in the oral cavity. The fluorophore interested is the flavin adenine dinucleotide (FAD), which in its oxidized form responds to a light of 450nm (blue-violet) emitting a fluorescence wavelength around 515nm. In cancer cells tissue fluorescence of the oxidized FAD irradiated with light blue-violet decreases due to alterations in the metabolism of the affected cells. Accordingly the G.o.c.c.i.e.s. has to be used with its compatible light source "G.o.c.c.i.e.s. light" that is characterized by an emission range of 440-490 nm including the excitation peak of FAD.

CONTRAINDICATION

No limitation or contraindication of use is known.

Attention! Before conducting the examination you must ensure that the patient has not ingested substances that can alter the effect of fluorescence (eg: assumption of licorice, coffee, other pigments)

NON-CONFORMING USE

The device must not be modified in its structure. This may lead to erroneous results, safety risk and the damage of the device.

The device may only be used according to the described conditions. Different uses must be authorized in writing by PIERREL PHARMA srl

WARRANTY

The warranty is void when:

- The instructions for use are not observed as shown
- The device is not used in the described uses
- There is alteration to or demounting of the unit
- There is damage of mechanical origin and damage to the environment, natural wear and tear
- Placement or structure is not appropriate.

STORAGE

- Store and use it at temperature range 5°C/40°C
- Store in a cool and dry place, far from UV source to avoid the degradation of the product.

ATTENTION

Read the instruction carefully before use

STAFF TRAINING

For the use and maintenance of the product, the specialized medical staff must apply and observe the instructions for use.

CHECKING AT THE DELIVERY

Check the packaging immediately upon delivery and the device during unpacking for any visible damage.

INSTRUCTION

The device has to be worn in adherence to the face, avoiding external light to reach the operator's eyes. Use the provided neck cord to optimize the fitting. Conduct the intra-oral examination using Goccles® eyewear in combination with a dental curing light with the following characteristics

- emission spectrum between 380 and 500 nm,
- emission peak between 440 and 480 nm.

CLEANING

The article can be cleaned using running water and dried with a soft non-abrasive cloth; lenses with anti-fog coating should only be cleaned using a soft cloth.

For disinfection with specific products, contact Pierrrel Pharma Srl

DISPOSAL

Follow the national regulations for the disposal of municipal waste. The product does not contain hazardous substances.

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Para la desinfección con productos específicos, póngase en contacto con Univet Srl.