

0 PREVIOUS EVALS
10 YEARS EXP

[The Mighty Mini 90] removed the stain without any added force/pressure.

Dana Jones, RDH
Dorchester, MA



3 PREVIOUS EVALS
36 YEARS EXP

I thought I was using the best angle available but this angle was so much better!

Linda Bell, RDH
Chester, NJ



1 PREVIOUS EVALS
39 YEARS EXP

[The Mighty Mini 90] reduces splatter even when the tooth isn't completely dry.

Susan Afeltra, RDH
Aurora, IL

PAC-DENT

Mighty Mini 90

4.5



The disposable prophylaxis angle combines optimal reach and visibility with the safety and convenience of out-of-the-pack sterility.

In a perfect world, prophylaxis angles would offer the reach, visibility, and power needed to ensure optimal cleaning and polishing—all while enhancing comfort and safety. Pac-Dent's version of this perfect world is the Mighty Mini 90 disposable prophylaxis angle (DPA). "[It's] small enough to reach difficult areas, and the outside of the cup works great at removing stains on the lingual surfaces," said Laura Inman, RDH, one of the hygienists who used the prophylaxis angles in her practice for this Dental Product Shopper evaluation, awarding it a Best Product rating.

Designed with a smaller head and ultra-slim profile for exceptional access and visibility, the Mighty Mini prophylaxis angles are available in non-sterile or pre-sterilized versions. Our evaluators tested the sterile angle. Praising the small cup size and ease of manipulation hygienist Amber Cutshall said, "I love that this prophylaxis angle is considered sterile."

Cleaning and Polishing Efficiency

With a small and powerful head, exceptional handling, performance-proven gear, and drive shaft technology—and what Pac-Dent calls the slimmest profile on the market—the Mighty Mini 90 DPA ensures optimal cleaning and efficient polishing. Citing the slim handle and ribbed cup sides as her favorite features, Amy Howard, RDH, said the Mighty Mini 90 provided great stain removal benefits. Dana Jones said the firm cup had proper detailing to remove plaque and stain quickly. In a patient with heavy interproximal tobacco staining, Virginia Kilduff was impressed that she could angle the cup mesial while the outside ridges were being used on the distal of the adjacent tooth



Several evaluators commented on ease of use, including Kathy Sweet, a hygienist who said the product made polishing easier and more proficient. Jones provided similar feedback, saying, "I didn't have to work as hard removing stain so polishing was more pleasant," and explained that the Mighty Mini 90 removed the stain without any added force or pressure. Maryann Beltz, RDH, suggested more stain-removing ridges inside the cup as a possible improvement. When asked which feature she liked most, Malee Wong said they are currently the only sterile prophylaxis angles available for purchase.

Improved Access and Visibility

The Mighty Mini 90's 20% shorter head provides superior clearance and reach, and the slimmer neck profile offers an unobstructed view of the work area, according to Pac-Dent. Noting that the Mighty Mini 90s proved critical in patients with limited openings, Linda Bell, RDH, said, "The mini head made access so much easier [and] allowed for better contouring on posterior molars." Beltz pointed out that the product worked well for children and small mouths, and Susan Afeltra found

takeaways

- Sterile right out of the pack
- A shorter head and slimmer profile provide optimal access and visibility
- Performance-proven gear and drive shaft technology ensure cleaning and polishing efficiency
- Operates smoothly with less splatter, noise, and vibration



Laura Inman, RDH
Clyde, NC

[IT'S] SMALL ENOUGH TO REACH DIFFICULT AREAS AND THE OUTSIDE OF THE CUP WORKS GREAT AT REMOVING STAINS ON THE LINGUAL SURFACES!

that the prophylaxis angle flexed well in areas where teeth were misaligned/crowded.

Sweet wanted a "smaller head to work more easily around teeth and a soft head to get closer on teeth and under the gumline," and concluded that the Mighty Mini 90 did both. She went on to explain that it was especially helpful for an autistic patient with a large, active tongue, and that it was much easier to get along the lingual of mandibular teeth. Jones noted that her patients seemed more comfortable, explaining, "With a larger head, patients seem to feel the head tickles their gingiva, especially on the palate. I found this happened less with this product."

Among the evaluators who commented on improved visibility were Howard, who said she experienced increased visibility with a 3-year-old's first dental visit, and Cutshall, who said, "I enjoyed how small the head of the product was, allowing for the best visibility possible." Cutshall went on to say, "I feel I was able to see more clearly in hard-to-access areas so I feel my ergonomics were improved."

Reduced Splatter and Vibration

The Mighty Mini 90's reduced splatter and quiet, sustained power prompted comments by a number of the evaluators. "The product held the prophylaxis paste well,

thus reducing splatter," noted Afeltra, who said the Mighty Mini 90 operates smoothly and reduces splatter even when the tooth isn't completely dry. While Kilduff still noticed some splatter and said this is an issue with all prophylaxis angles, hygienist Tammy Sjurseth told DPS, "I really like how smoothly the polisher worked with little splatter." Bell said the ribbed cup reduces splatter and commented, "[I] absolutely love this angle."

colleagues, and collectively the group deemed it a Best Product. "I thought I was using the best angle available, but this angle was so much better," Bell announced. And Inman concluded, "I have used [other] prophylaxis angles religiously for the past 10 years and have never been able to find a prophylaxis angle that would surpass [them] until now. I have already switched to this brand."

Overall Satisfaction


All of the evaluators said they would recommend the Mighty Mini 90 to

FOR FREE INFORMATION:
800.921.4806 ext. 124
www.dps.li/a/5S7-XXX
Circle XXX on the card



PRODUCT EVALUATION SNAPSHOT

CRITERIA BASED ON AVERAGE SCORE (OUT OF 5)

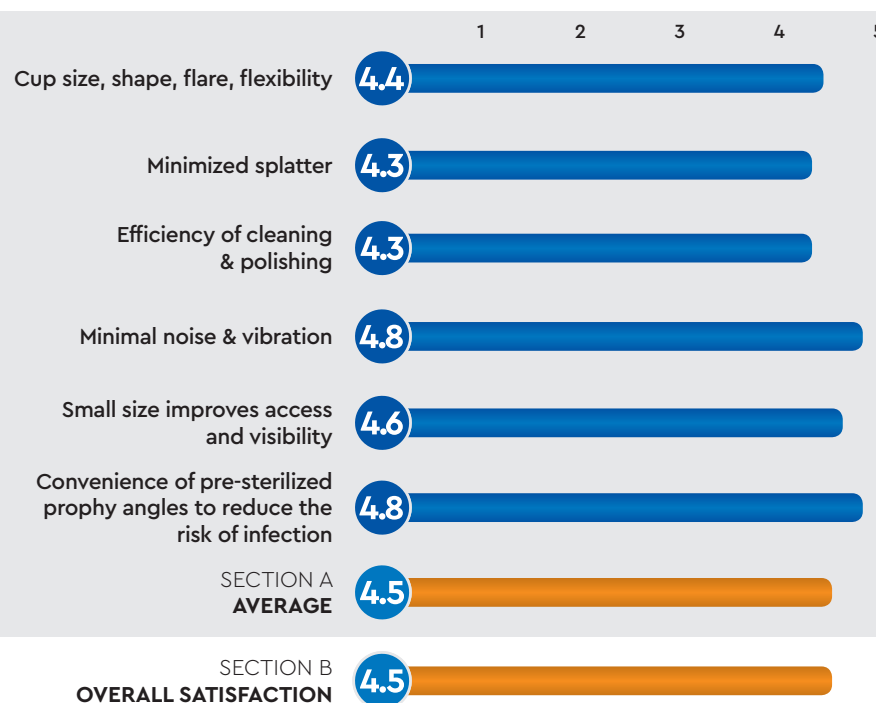

PAC-DENT
MIGHTY MINI 90

FINAL SCORE
AVERAGE OF SECTIONS A AND B

4.5

