Lucitone 199® Denture Base Disc

Directions for Use
Mode d'emploi
Gebrauchsanweisung
Istruzioni per l'uso
Instrucciones de uso

 Responsible Manufacturer:
Dentsply Prosthetics
570 West College Avenue
York, PA 17401 U.S.A.
Tel. 1-800-243-1942
dentsplysirona.com

EC REP
DeguDent GmbH
Rodenbacher Chaussee 4
63457 Hanau-Wolfgang
Germany
Tel. +49 (0) 6181-5950

R^{only} CE
0086

DP-0000190 Rev. 2 (01/18)

THE DENTAL
SOLUTIONS
COMPANY™

 **Dentsply
Sirona**

en

Product Description:

Lucitone 199® Denture Base Disc is intended for use in milling full and partial denture prosthodontic appliances. The discs are available in 98.5mm diameter to fit most open architecture, 5-axis milling machines. Discs are available in 4 heights; 20mm, 25mm, 30mm and 35mm.

Indications:

Lucitone 199® Denture Base Materials are intended for the fabrication of PMMA based acrylic dental prostheses including but not limited to partial and full dentures.

Contraindications:

Lucitone 199® is contraindicated for patients who have a known hypersensitivity or severe allergic reactions to any of the components.

Warnings:

Lucitone 199® contains residual methyl methacryl monomer which may cause skin sensitization (allergic contact dermatitis) or other allergic reactions in susceptible persons. If skin sensitization occurs, discontinue use. If dermatitis or other symptoms persist, seek medical advice.

Adverse Reactions:

Particulates will be generated when milling or grinding acrylate resins. Safety glasses and / or gloves are recommended when milling the discs to minimize the potential for eye, skin and respiratory irritation. Adverse effects from use of this medical product are highly unlikely with proper handling and processing. However, reactions of the immune system (such as allergies) to substances contained in the material or localized paraesthesia (such as taste disturbances or irritation of the oral mucosa) cannot be completely ruled out.

Precautions:

The following safety and warning information describes how to use the Lucitone 199 Denture Base Disc in a safe and risk-free manner.

- Keep product dust away from eyes.
- Do not inhale dust particles during grinding.
- Use local vacuum suction and suitable mouth/face protection during manual machining at the workplace.
- If ingested contact regional Poison Control Center.
- Comply with pertinent SDS (safety data sheets)

Body Label:

The body label provides key product information. The specific disc characteristics height and shade are listed along with the lot number, manufacturing date and shelf life date.

Dentsply Sirona Lucitone 199® Denture Base Disc **LOT** 00000000000 **Exp. Date** YYYY-MM-DD **R^{only} CE** 0086
Height: 20 mm **Shade:** Original **Mfg Date** YYYY-MM-DD **REF** 906030 **Exp. Date** 4429-A

Use Instructions:

Lucitone 199 Denture Base Discs are available in 98.5mm diameter. This dimension will fit the most popular 5-axis milling machines. The discs are recommended for the production of full or partial denture upper and lower arches for bonded digital dentures.

1. Mount the disc in the milling machine jig following the equipment manufacturer's instructions.
2. After milling, remove the disc from the machine and remove the sprues. Minimal finishing of the denture base may be necessary. Follow your laboratory's recommended technique for finishing, festooning, and polishing dentures.
3. In order to finish the denture pre-manufactured teeth need to be bonded to the denture base.

Dentsply Sirona recommends Portrait® IPN® teeth.

Important Considerations:

- > Avoid heat and light during storage
- > Single use only
- > Use care and judgment to avoid generating high heat during milling and polishing, as it may damage the denture base.
- > Do not use autoclave to sterilize

Symbols on product labels:

- LOT** Batch Code
- REF** Reorder Number
-  Manufacture Date
-  Expiration date
-  Consult (electronic) instructions for use

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