Prior to use, carefully read the instructions for use. EN

G-aenial™ Flo X LIGHT-CURED RADIOPAQUE FLOWABLE COMPOSITE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- 1. Liner or base
2. Combining of restorations
3. Repair of (in) direct aesthetic restorations, temporary crown & bridge, defect margins when margins are in enamel
4. Sealing hypersensitive areas
5. Fissure sealant
6. Direct restorative for Class II, III, IV, I and V cavities including small cavities which require high flowability.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

Operation

- 1. Preparations
2. Shade Selection
3. Cavity Preparation
4. Bonding
5. Placement of G-aenial Flo X

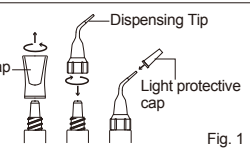
Fig.1

- 1. Material should be placed and light cured in layers. For maximum layer thickness, please consult the table.
2. Lower light intensity may cause insufficient curing or discoloration of the material.
7. Finishing and Polishing
Finish and polish using standard techniques.

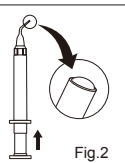
Fig.2

Clinical Hint
In order to inject effectively, use the surface tension of the material to ensure uniformity across the entire surface of the restoration during build up. Once the required amount has been injected, release the pressure on the plunger and withdraw the syringe in a direction perpendicular to the surface.

Table with 3 columns: Shade, Irradiation time, 10 sec. (High power LED) (1200mW/cm²), 20 sec. (HaloGen/LED) (700 mW/cm²)

- 1. Material should be placed and light cured in layers. For maximum layer thickness, please consult the table.
2. Lower light intensity may cause insufficient curing or discoloration of the material.

Table with 3 columns: Shade, Irradiation time, 10 sec. (High power LED) (1200mW/cm²), 20 sec. (HaloGen/LED) (700 mW/cm²)

STORAGE
Recommended for optimal performance, store in a cool and dark place (4-25°C / 39.2-77.0°F) away from high temperatures or direct sunlight.

- 1. Sprinkle 3.6g (2.0mL), 20 dispensing tip III (plastic type), 1 light protective cap
2. 30 dispensing tip II (plastic type), 2 light protective caps
3. 30 dispensing tip III (needle type), 2 light protective caps

- 1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. Take care to avoid ingestion of the material.
4. Do not mix with other similar products.
5. Dispensing tips are for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilized in an autoclave or chemically.

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Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gcaerica.com
or for The Americas
http://www.gcomerica.com
They can also be obtained from your supplier.

CLEANING AND DESINFECTING:
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE: Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised: 06/2018

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Tormitsu-cho, Kasugai, Aichi 486-0844, Japan

EU : GC EUROPE N.V.
Researchpark Haaxrode-Laanen 1240, Breda, Belgium 33.
B-3020 Leuven, Belgium. TEL: +32 16 76 14 00
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60603 U.S.A.

DESIGNED BY:
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8588, Japan

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60603 U.S.A.
TEL: +1 708-597-0900
http://www.gcaerica.com

GC SOUTH AMERICA
Rua Heliodora, 369, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.272.999/0001-61
RESP. TEC: Mayara de Santos Ribeiro - CRO/SP 105.182

GC ASIA DENTAL PTE. LTD.
11 Tines Concourse, #03-05 Singapore 828729
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Bankmeadow, NSW 2019, Australia
TEL: +61 2 9301 6200

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