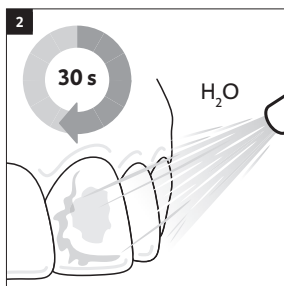
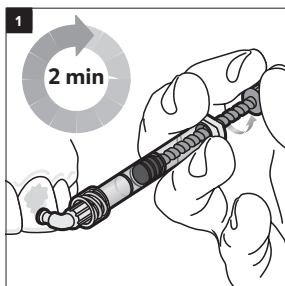
Icon-Etch

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English

Product description

The etching gel Icon-Etch forms part of infiltration treatment with Icon. For the entire infiltration treatment, the following instructions for use must be strictly observed:

- Icon smooth surface

Intended purpose

Removing the pseudointact surface layer in connection with infiltration treatment with Icon.

Limitations of use

Sufficient drying and protection of the gingiva not possible.

Indications

- Non-cavitated carious lesions
- Fluorosis

Contraindications

Do not use this material:

- in cases of allergies to any of the ingredients or in cases of contact allergies.

Patient target group

People treated in the course of a dental procedure.

Intended users

Dentist

Notes for use

- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must only be used on one patient for hygiene reasons.
- The etched surface must not be touched or contaminated with saliva or moisture until the treatment resumes. If contamination occurs after drying re-etch for approx. 10 seconds and

repeat drying with Icon-Dry.

- For best treatment results a sufficiently dry working area is critical. Therefore, appropriate measures for a completely isolation must be taken. A liquid dam can be used as well. Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).

Recommended use

Application of Icon-Etch (for image, see Handling):

- Screw the Smooth Surface-Tip onto the Icon-Etch syringe.
- Apply an ample amount of Icon-Etch onto the lesion site by turning the syringe shaft carefully, let set for 2 minutes and activate it by moving (Fig.1). Remove excess material with dental wadding.
- Suction off Icon-Etch and rinse with water for at least 30 s. Then dry using oil-free and water-free air (Fig. 2).

Interactions

Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).

Residual risks/side effects

- There are no known side effects to date. The residual risk of hypersensitivity in relation to allergies to components of the material cannot be ruled out.
- Contact of Icon-Etch with the oral mucosa causes a white surface coloration. This will subside after a few days.

Warnings / precautions

- Keep out of the reach of children.
- Caution! Icon-Etch can cause chemical burns
- Avoid contact with soft oral tissue, eyes, and skin. In case of inadvertent contact rinse with copious amounts of water, and, if necessary, consult a physician.
- Limit the application to the surface to be treated. Avoid contact of the etching gel with the gingiva, exposed dentine, root element and adjacent teeth.
- Use only the enclosed syringe and application tip to apply the etching gel in the patient's mouth.
- Dispose of application tip after use. Reattach syringe closure.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

pH value ≤ 0

Storage / disposal

- Store at 2-25 °C / 36-77 °F!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

Composition

Hydrochloric acid, pyrogenic silicic acid, surface-active substances

Note: A glossary of all symbols displayed is at: <http://www.dmg-america.com/literature>

Caution: U.S. Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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