



Chairside Guide




Prepare cavity
Bevel enamel margins

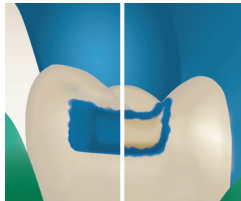

Total or selective etch

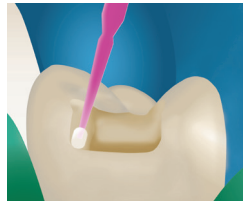

Apply bonding agent
per instructions

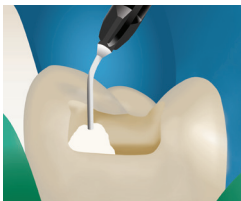

Place Presto in 2mm
increments



Light cure after
each layer



Finish and polish

Ensuring Successful Composite Restorations

Instructions

- Follow manufacturer's instructions for each material and device you are using.

Biofilm

- Air abrade or pumice tooth to remove biofilm prior to etching and bonding.

Curing Light

- Clean light tip.
- Check light output with radiometer.
- Place barrier sleeve so seam does not obstruct light source.
- Position light perpendicular and close to restoration.
- Watch target using eye protection while light curing.



80 Oakland Street • Watertown, MA 02472 • U.S.A.

Tel. (617) 926-6666 / (800) 343-4342 / Fax (617) 926-6262 • sales@pulpdent.com • www.pulpdent.com