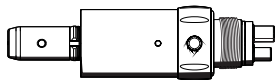
# Midwest<sup>®</sup> Rhino XE

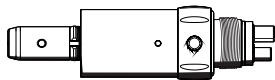
English / Français / Español

Operating instructions  
Notice d'utilisation  
Instrucciones de uso

# Midwest<sup>®</sup> Rhino XE

English

## Operating instructions

# Table of Contents

<b>1</b>	<b>Before you begin</b> .....	<b>5</b>
1.1	Structure of the document .....	6
1.2	Service life .....	9
<b>2</b>	<b>Safety instructions</b> .....	<b>10</b>
<b>3</b>	<b>Technical description</b> .....	<b>13</b>
3.1	Task .....	13
3.2	Structure of Midwest Rhino XE .....	13
3.3	Technical Data .....	14
3.4	Product labeling .....	16
<b>4</b>	<b>Preparation</b> .....	<b>18</b>
4.1	Initial start-up and longer breaks in use .....	18

<b>5</b>	<b>Operation</b> .....	<b>19</b>
5.1	Connect motor to the supply hose .....	19
5.2	Replacing the instrument .....	20
5.3	Setting drill direction on the motor .....	21
<b>6</b>	<b>Postprocessing</b> .....	<b>22</b>
6.1	After each treatment session .....	22
6.2	At the end of the work day .....	23
6.3	Once per work week .....	23
<b>7</b>	<b>Reprocessing</b> .....	<b>24</b>
7.1	Automated cleaning .....	24
7.2	Manual cleaning and disinfection.....	24
7.3	Manual cleaning.....	26

	7.4	Manual maintenance .....	28
	7.5	Sterilizing .....	30
<b>8</b>		<b>Maintenance .....</b>	<b>33</b>
	8.1	Replacing the sealing washer .....	33
	8.2	Lubricate the lock washer .....	34
<b>9</b>		<b>Spare parts and consumables .....</b>	<b>35</b>
<b>10</b>		<b>Disposal .....</b>	<b>36</b>

## 1 Before you begin ...

The motor complies with the regulations reflecting the current state of technology. The motor complies with the ISO 14457 standard.

1. Read the operating instructions prior to using the Midwest Rhino XE motor.
2. Use the motor only for applications that are described in the operating instructions.
3. Follow the hygiene regulations, work safety regulations, and accident prevention measures that apply to the motor in your country.

Low-speed (vane motor) handpiece is used for drilling of teeth in a variety of dental procedures.

none

This product is intended only for use by trained dental personnel in dental practices and laboratories.

**Intended use**

**Contraindications**

**Target group**

## Warnings

## Instructions for use

### 1.1 Structure of the document

#### 1.1.1 Labeling of information

- To prevent injuries, please observe the warnings.

Warnings are labeled as follows:

**⚠ DANGER!** indicates a danger **leading** to death or serious injury if not avoided.

**⚠ WARNING!** indicates a danger that **may lead** to death or serious injury if not avoided.

**⚠ CAUTION!** indicates a danger that **may lead** to injury if not avoided.

- To prevent material damage and additional expenses, please observe all instructions for use.

Instructions for use are labeled as follows:

**NOTE!** indicates measures for the prevention of material damage.

**IMPORTANT:** indicates information on the avoidance of additional expenses and other important information.

**Tip:** indicates information for facilitating work.

### 1.1.2 Formats and symbols

The formats and symbols used in this document have the following meaning:

<ul style="list-style-type: none"> <li>✓ Prerequisite</li> <li>1. First action step</li> <li>2. Second action step</li> <li>or</li> <li>➤ Alternative action</li> <li>↪ Result</li> <li>➤ Individual action step</li> </ul>	Requests you to do something.
Use of formats and symbols [→ 8].	Identifies a reference to another text passage and specifies its page number.
<ul style="list-style-type: none"> <li>• List</li> </ul>	Identifies a list.

## 1.2 Service life

When used as intended:

- Moving parts have a typical service life of 1 year.

No warranty claim can be inferred here, as wear may occur earlier or later than indicated above depending on use, frequency of sterilization, and frequency of maintenance.

## Obligations of the user

## Preventing the spread of infections and cross contamination

## 2 Safety instructions

- Use only fault-free materials that do **not** deviate from the specified data [→ 14].
- Protect yourself, patients, and others from danger. To do this, observe the safety information.
- Comply with the Intended use of the equipment.
- You should always keep these operating instructions within reach for further reference.

Prevent the spread of infections and cross contamination between patients, users, and third parties. Sterilize equipment after each patient.

Take the appropriate hygiene measures, e.g. wear protective gloves.

## Hot motor

## Malfunction or damage

## Repair

## Spare and accessory parts

## First aid measures in the case of lubricant accidents

If the motor is defect, it may get hot.

Discontinue use immediately in case of malfunction, unusual or different sounds or damage. Damaged motors may cause injury. Notify the dental depot or the manufacturer.

Do **not** repair the motor yourself.

Use only original Midwest parts or parts approved by Midwest. Safe operation is **not** guaranteed for parts that have not been approved by Midwest.

### General information:

Immediately remove any clothing soiled by the product.

### After inhalation:

Supply fresh air; consult doctor in case of complaints.

### After skin contact:

If skin irritation continues, consult a doctor.

**After eye contact:**

Rinse opened eye for several minutes under running water.

**After swallowing:**

If symptoms persist consult doctor.

For details download the Midwest Plus Spray Material Safety Data Sheet from the Midwest homepage: [www.dentsplysirona.com](http://www.dentsplysirona.com)

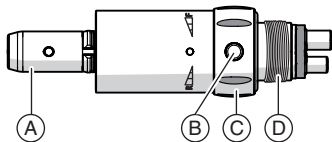
If you have any questions, please contact your dental supplies distributor or the manufacturer.

## 3 Technical description

### 3.1 Task

The motor converts air to rotation and transfers rotation to the instrument.

### 3.2 Structure of Midwest Rhino XE

A	Handpiece holder
B	Drill direction marking
C	Drill direction switching ring
D	Coupling connection (here: Midwest/ISO 4-hole)

### 3.3 Technical Data

#### Air motor

	Midwest Rhino XE
Motor speed in rpm	0 - 24 000 ± 20%
Torque in N/cm	≥ 1.5
Handpiece holder – ISO 3964	x

**Dental treatment center**

	<b>Midwest Rhino XE</b>
Driving air pressure in bar (psi) (flowing, dry, and clean)	$2.9 \pm 0.2$ ( $42 \pm 2.9$ )
Return air pressure in bar	$< 0.15$
Air consumption in NI/min	$\leq 80$

### 3.4 Product labeling

Sterilizable in a steam sterilizer (autoclave) at the temperature specified

Date of manufacture

Serial Number

Order number

Data matrix code (here: an example)

Content of the data matrix code:

+E2768752050/\$\$+5192/16D20170309E

A

B

C

D

- A Manufacturer code (here: E276)
- B REF number (here: 875205)
- C Serial no. (here: 5192)
- D Manufacture date (YYYYMMDD)

## 4 Preparation


### 4.1 Initial start-up and longer breaks in use

- Sterilize Midwest Rhino XE and accessories prior to startup.
- Clean and maintain Midwest Rhino XE after longer breaks in use.

## 5 Operation

### 5.1 Connect motor to the supply hose

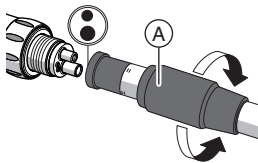1. Attach the motor to the supply hose, noting the various tube diameters.
2. Screw the screw cap (A) hand-tight. Hold the motor securely when doing this.

Does water/air leak out between the motor and the hose connection?

1. Detach the motor from the supply hose.
2. Reconnect the motor to the supply hose. Make sure it is connected properly.
3. If water/air still leaks out: replace the sealing washer [→ 33].

## 5.2 Replacing the instrument

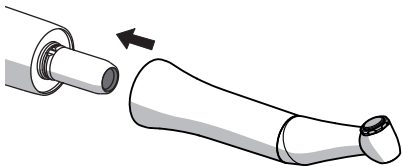
**⚠ CAUTION!** The instrument should only be fitted or removed when the motor is at standstill.

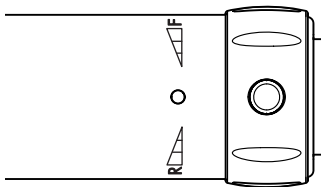
### Attaching the instrument

- ✓ The motor is at a standstill.
- Insert the instrument until it snaps into place.

### Removing the instrument

- ✓ The motor is at a standstill.
- Detach the instrument. Do not pull on the supply hose while doing this.

### 5.3 Setting drill direction on the motor

The motor has two drill directions:

- Clockwise (forward), marked F
- Counterclockwise (reverse), marked R

#### Set forward

- Turn the switching ring clockwise to the stop.

#### Set reverse

- Turn the switching ring counterclockwise to the stop.

**IMPORTANT!** If the marking is in the middle between R and F, the motor is switched off.

## 6 Postprocessing

### 6.1 After each treatment session

**NOTE!** Condition immediately, or at the latest, one hour after treatment.

**NOTE!** Use only Midwest Plus Spray.

**NOTE!** The air motor is **not** suitable for use with a cleaning and disinfection device.

- ✓ The burr instrument must not be moving.
- ✓ Wear appropriate protective clothing.
- 1. Remove the burr instrument with tweezers.
- 2. Detach the instrument from the motor.
- 3. Transport the motor to the hygiene room in a suitable transport container.

4. Conduct a manual reprocessing. [→ 24] Observe the relevant national/local regulations.
5. Apply spray to the motor [→ 28].
6. Sterilize the motor [→ 30].

## 6.2 At the end of the work day

- Apply spray [→ 28] to Midwest Rhino XE.

**NOTE!** Do not leave any instruments on the motor overnight, in order to prevent oil from leaking into the air motor.

## 6.3 Once per work week

- ✓ The motor has been conditioned.
- Lubricate the lock washer [→ 34].

## 7 Reprocessing

### 7.1 Automated cleaning

**NOTE!** The air motor is **not** suitable for use with a cleaning and disinfection device.

### 7.2 Manual cleaning and disinfection

**IMPORTANT:** Manual reprocessing is possible in exceptional cases if the national/local regulations are followed. The national/local regulations are to be checked before.

**NOTE!** Condition immediately, or at the latest, one hour after treatment.

**NOTE! Never** clean in an ultrasonic bath!

**NOTE! Never** immerse in disinfectants!

**NOTE!** Use only Midwest Plus Spray.

**IMPORTANT:** Use a soft, clean, and disinfected brush for cleaning.

### 7.3 Manual cleaning

**IMPORTANT:** Observe the respective national / local requirements for manual reprocessing.

**NOTE!** Condition immediately, or at the latest, one hour after treatment.

**NOTE! Never** clean in an ultrasonic bath!

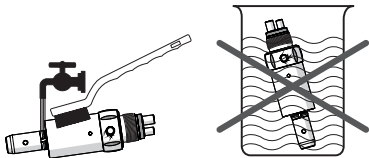
**NOTE! Never** immerse in disinfectants!

**NOTE!** Use only Midwest Plus Spray.

**IMPORTANT:** Use a soft, clean, and disinfected brush for cleaning.

**IMPORTANT:** The Midwest Handpiece Cleaner is a surfactant.

✓ Wear appropriate protective clothing.

1. Spray a sufficient amount of Midwest Handpiece Cleaner onto a clean, lint-free cloth and use it to clean the entire external surface of the instrument.
2. Brush Midwest Rhino XE under running water (< 38 °C, < 100 °F, at least drinking water quality) and good lighting (min. 500 lux) and color rendering index (min. 80 Ra) until no more dirt can be seen, for at least 10 seconds.
3. Flush the drive channels with spray.
4. Lubricate mechanical parts manually [→ 28].
5. Pack Midwest Rhino XE in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
6. Conduct sterilization [→ 30].

## 7.4 Manual maintenance

### 7.4.1 Maintenance of mechanical parts

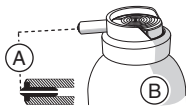
#### Intervals

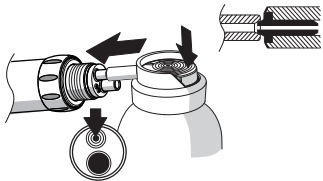
- After each patient
- Prior to each sterilization

#### Required accessories

- A Black nozzle insert
- B Midwest Plus Spray

**NOTE!** Use only Midwest Plus Spray.

## Procedure

- ✓ The instrument is removed [→ 20].
  - ✓ The black nozzle insert is in the spray can tube.
1. Detach the motor from the supply hose [→ 19].
  2. Set the drill direction to F.
  3. Insert the nozzle of the spray can in the driving air tube. Hold the motor with a cloth.
  4. Spray into the motor for 1-2 seconds.  
**IMPORTANT:** Hold the spray can upright.
  5. Wipe any spray that comes out with a disinfection cloth.
  6. Repeat the process until the spray is clear.

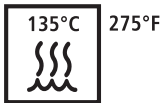
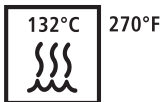
## 7.5 Sterilizing

### Intervals:

- Prior to initial operation
- Prior each subsequent use

### Procedure

- ✓ Midwest Rhino XE is clean.
- ✓ Midwest Rhino XE maintenance is complete.
- ✓ If necessary, Midwest Rhino XE can be packed in packaging suitable for sterilization and storage, e.g. paper/plastic composite packaging or container.
- Sterilize Midwest Rhino XE in the steam sterilizer with saturated water vapor.

Temperature: 132 °C (270 °F) / 135 °C (275 °F)

Overpressure: 1.87 bar (27.12 psi) / 2.13 bar (30.89 psi)

Wrapping	Steam sterilization	Holding time	Drying time
Wrapped	Gravitation process	15 minutes at 132 °C	30 minutes
Wrapped	Pre-vacuum process	4 minutes at 132 °C	30 minutes
Wrapped	Pre-vacuum process	3 minutes at 135 °C	16 minutes

**NOTE!** Do not exceed 140 °C (284 °F), even during the drying phase.

#### After sterilizing

1. Remove Midwest Rhino XE from the steam sterilizer immediately.

**⚠ CAUTION!** Midwest Rhino XE is hot. Risk of burns!

**NOTE!** Do **not** try to accelerate the cool-down process by placing it in cold water. This will damage your Midwest Rhino XE!

2. Store Midwest Rhino XE so that it is protected against contamination.
3. Sterilize again once the storage period has elapsed.

## 8 Maintenance

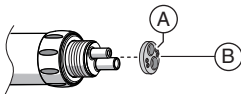
Elastomers, e.g. O-rings, must be replaced depending on their degree of wear.

### 8.1 Replacing the sealing washer

If water/air leaks out between the motor and the hose connection, replace the sealing washer.

1. Disconnect the motor from the supply hose.
2. Use a probe etc. to remove the defective sealing washer (B) from the rear of the motor.
3. Attach the new sealing washer, observing the position of the tubes and contact pins and press it to the stop.

**IMPORTANT:** The bead (A) on the sealing washer faces the supply hose.


## 8.2 Lubricate the lock washer

### Interval

- 1x weekly

### Procedure

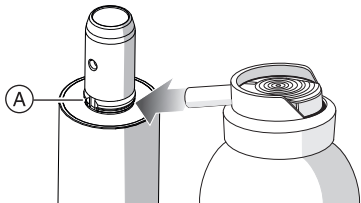1. Spray some T1 spray on the stop spring ring (A).
2. Turn the stop spring ring to distribute the T1 spray.

## 9 Spare parts and consumables

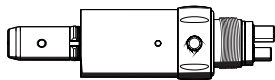
Use only original Midwest parts or parts approved by Midwest.

	REF		REF
Midwest Plus Spray (6 cans, 250 ml each)	38 00 80M	Midwest gasket	58 55 460

## 10 Disposal

- According to current information, the product does not contain any substances that are hazardous to the environment.
- Disinfect the product prior to disposal.
- Observe the applicable disposal regulations for your area.

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**Sirona Dental Systems GmbH**