

G2-BOND Universal TECHNIQUE GUIDE



DISPENSING INTO DISPOSABLE DISPENSING DISH (BOTTLE)



Shake the bottle and dispense 1-PRIMER into "1" in disposable dispensing dish.

Dispense 2-BOND into "2" in disposable dispensing dish.

Use of G2-BOND Universal bottle covers is highly recommended. Bottle covers are designed to protect product from body heat, which might impact product working time. The bottle covers may be reused with subsequent refills. When bottle is empty, remove cover and clean with water or wipe with alcohol. Do not sterilize, autoclave, or clean with chemical disinfectants.

ETCHING MODE SELECTION



Etch for 10-15 sec., rinse for 5 sec. and dry.

Light curing time

Light curing unit	Distance from light guide tip	
	<10mm	>10mm
Halogen / LED (700-1200 mW/cm <sup>2</sup> )	10 sec.	20 sec.
High power LED (more than 1200 mW/cm <sup>2</sup> )	5 sec.	10 sec.

G2-BOND Universal TECHNIQUE GUIDE



BONDING PROCEDURE (INDICATIONS FOR USE [1])



INTRAORAL REPAIR (INDICATIONS FOR USE [3] [4])



SEALING OF TOOTH PREPARATION (INDICATIONS FOR USE [6])

