



Instructions for Use

Cleaning:

- Instruments must be cleaned and sterilized prior to each use (including the first use).
- Automated cleaning instructions using a washer-disinfector:
- Recommended cycle:
 - 2-minute pre-cleaning with cold tap water; Drain
 - 5-minute cleaning with 55°C/131°F cleaning solution; Drain
 - 3-minute rinsing with cold deionized water; Drain
 - 2-minute rinsing with cold deionized water; Drain
 - Allow instruments to fully dry.

Sterilization:

- Place Instruments in a sterilization pouch or cassette and load into autoclave:
 - Autoclave for 4 minutes at 134°C/273°F or 20 minutes at 121°C/250°F
 - Steam sterilization with a fractioned pre-vacuum is recommended.
 - Allow autoclave to complete the drying cycle.

Manufactured for:
Patterson Dental Supply, Inc.
1031 Mendota Heights Road
Saint Paul, MN 55120