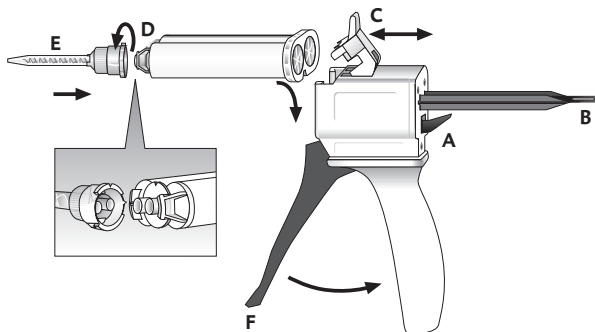
# TempoCem

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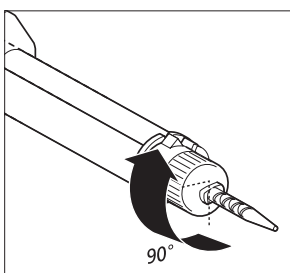
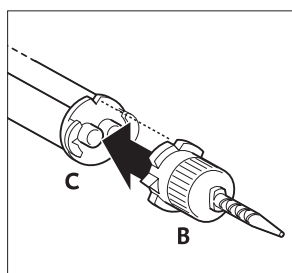
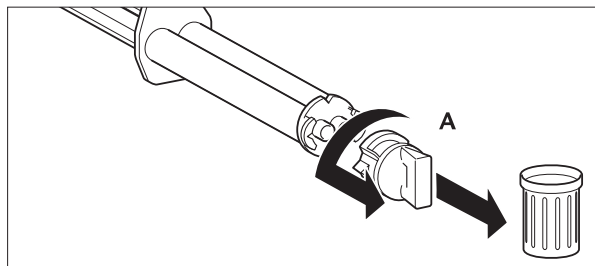
English

### Product description

TempoCem is a cement based on zinc oxide and eugenol and is automatically mixed. TempoCem flows easily, allowing correct placement of the temporary restoration. It is resistant to saliva and is stable in the mouth. It withstands the stress of mastication sufficiently, but allows easy removal of the temporary restoration.

### Indications for use

Temporary luting of crowns, bridges, inlays and onlays and implant-supported dentures.

### Limitations of use

Do not apply the material directly to the exposed dental pulp.

### Contraindication

Do not use the material where there are known allergies to any of the components or contact allergies.

### Patient target group

Persons receiving treatment as part of a dental procedure, for example:  
 Cementation of indirect restorations and prostheses for the treatment of large dental defects, tooth loss, malocclusions and malformations of the teeth.

### Intended users

Dentist

### Notes for use

- With the Smartmix version the material is applied directly without a dispensing device.
- With the Automix version the material is applied using a 1:1 dispenser.
- Cements containing eugenol can inhibit the curing of composites, resin-based luting cements and bonds.
- Check whether it might be more beneficial to use the eugenol-free luting cement TempoCem NE (DMG).
- Make sure that any residual cement is completely removed from the prepared tooth or core or, if applicable, from the implant abutment, following removal of the temporary restoration.

### Using the Automix Cartridge (for image, see »Handling«)

#### Use of the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift the plastic lever [C], insert the Automix Cartridge and lock it using the plastic lever [C].

**Note:** Make sure that the notches on the Automix Cartridge line up with those on the Automix Dispenser.

#### Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available in refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Twist the mixing tip clockwise by 90° to lock it.

#### Applying the material

- ▶ When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ Mix the material in the mixing tip using the lever [F] on the Automix Dispenser and apply directly.

**Note:** After the application, leave the used mixing tip on the Automix Cartridge as a seal!

#### Removal of the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic lever [C] and remove the Automix Cartridge.

### Using the Smartmix Syringe (for image, see »Handling«)

1. Before attaching the mixing tip, remove the cap [A] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [B].

**Note:** In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available in refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the Smartmix Syringe [C].

3. Twist the mixing tip clockwise by 90° to lock it.
4. The material is mixed in the mixing tip when ejected and can then be applied directly.

**Note:** When using a Smartmix Syringe for the first time, squeeze out a pea-sized amount of material and discard it!

**Note:** After application, leave the used mixing tip on the Smartmix Syringe as a seal!

## Disinfection of the Automix Dispenser

- The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear. Disinfection can be carried out using a commercially available dipping disinfectant. The Automix Dispenser is also autoclavable.
- Dirty or contaminated mixing tips used as seals can be cleaned with disinfectants normally used in the practice.
- DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

## Recommended use

### Temporary luting – General

1. Apply TempoCem to the dry temporary restoration.
2. Insert the temporary restoration within 60 seconds after mixing has begun and attach to the dry, prepared tooth using slight pressure.
3. After curing, remove excess cement with a probe. The excess cement can be easily removed in large pieces.
4. Remove interdental excess carefully using dental floss. The light shade of the material makes it easy to distinguish it from the teeth.

### Temporary luting of implant-borne dental prosthesis

1. Apply TempoCem to the superstructure.
2. Insert the superstructure within 60 seconds after mixing has begun and attach to the dry implant abutment using slight pressure.
3. After curing, remove excess cement with an implant scaler. Follow the manufacturer's instructions for the implant precisely!
4. Remove interdental excess carefully using dental floss. The light shade of the material makes it easy to distinguish it from the teeth.

## Timing

Working time (including mixing time)*	Max. 60 s
Maximum intraoral curing time (including mixing time)	420 s

\* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times.

## Interactions

Eugenol-containing materials can inhibit the polymerization of methacrylate-based materials.

## Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

## Warnings/precautions

- For dental use only!
- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.
- Contains eugenol, rosin.

## Product performance characteristics

Complies with ISO 3107:2011, Type 1

## Storage/disposal

- Store in a dry place at room temperature 15-25 °C (59-77 °F).
- Do not store in the refrigerator!
- Protect against exposure to direct sunlight!
- After application, leave the used mixing tip on the Automix Cartridge/ Smartmix Syringe as a seal!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

## Composition

Zinc oxide, rosin, hydrogenated polydecene, eugenol, additives.

**Note:** A glossary of all symbols is displayed at: <http://www.dmg-america.com/literature>

**Caution:** US Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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